MPC - FR

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The screenshot shows a mobile application interface for validating and sending data. At the top, there are navigation buttons: a blue arrow pointing left labeled 'Retour' and a blue 'X' icon labeled 'Envoi'. Below this is a section titled 'Valider et envoyer' with a subtitle 'Vous avez rempli 5 champs sur 25 et joint 0 photo'. There are three expandable sections, each with a dropdown arrow on the right: 1. 'Biochimie' with '5 champs sur 14' and a blue 'Modifier' button. 2. 'Paramètres cardiologiques' with '0 champs sur 2' and a blue 'Compléter' button. 3. 'Hématologie' with '0 champs sur 9' and a blue 'Compléter' button. At the bottom of the screen is a large blue button labeled 'Envoyer'. The bottom of the screen shows a grey bar with three small icons: a back arrow, a home circle, and a recent apps square.

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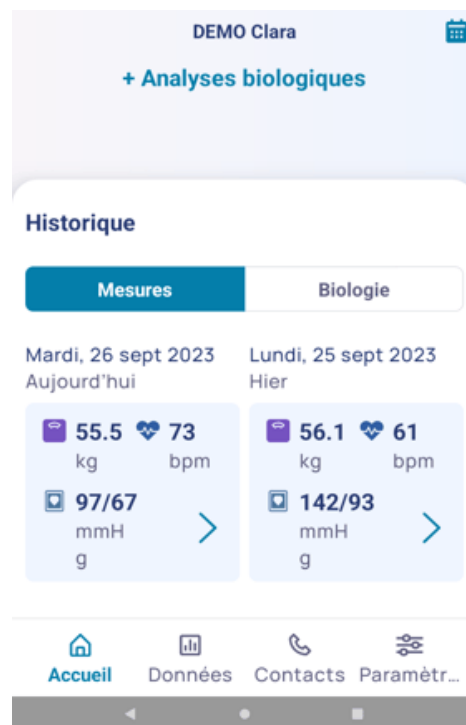
The screenshot shows a mobile application interface for reporting an absence. At the top, there is a navigation bar with a back arrow and the text 'Retour' on the left, and 'Absence' in the center. Below the navigation bar, the main heading reads 'Indiquez la raison de votre absence'. There are three selectable options, each with an icon and text: 1. 'Hospitalisation pour le cœur' with a heart and pulse icon. 2. 'Autre hospitalisation' with a white cross on a blue square icon. 3. 'Raison personnelle' with an icon of two people and a question mark. The bottom of the screen shows a standard Android navigation bar with back, home, and recent apps buttons.

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The screenshot shows the same mobile application interface, but at the date selection step. The navigation bar remains the same. The main heading now reads 'Veuillez saisir les dates de votre absence'. Below this, there are two date pickers. The first is labeled 'du' and shows the date '27/09/2023' with a calendar icon. The second is labeled 'au' and shows the date '04/10/2023' with a calendar icon. Below the second date picker, the word 'optionnelle' is written in small text. At the bottom of the screen, there is a large blue button with the text 'Envoyer mon absence'. The bottom navigation bar is also visible.

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The screenshot shows a mobile app interface for reporting symptoms. At the top, the status bar displays the time 16:03, signal strength, and battery level at 56%. The app header includes a back arrow labeled 'Retour', the title 'Étape 3/3 Symptômes', and a close icon. The main heading asks 'Par rapport à hier, quelle gêne nouvelle ressentez-vous ?' with a subtext '1 ou plusieurs réponses possibles'. Below this, there are six light blue buttons with radio button icons: 'Je tousse plus', 'Je suis plus essouffé', 'Je rajoute un oreiller pour dormir', 'Je suis plus fatigué', 'J'ai les chevilles plus gonflées', and 'Rien à signaler'. The bottom of the screen shows the standard Android navigation bar with three icons: a square, a circle, and a triangle.

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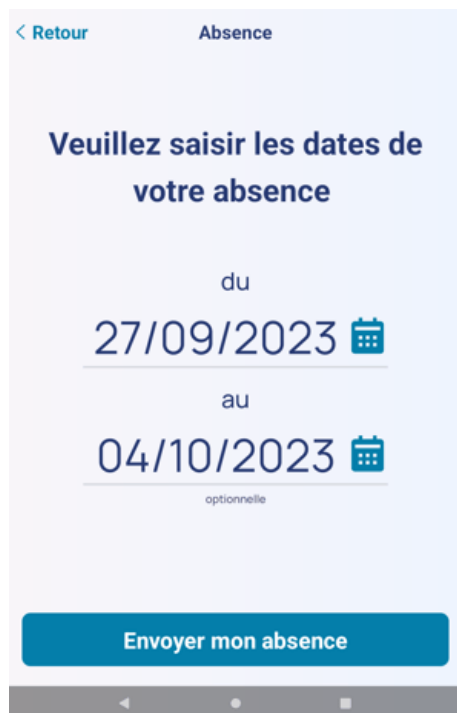
The screenshot shows a mobile application interface for reporting an absence. At the top, there is a navigation bar with a back arrow and the text 'Retour' on the left, and the title 'Absence' on the right. The main content area has a light blue background and contains the text 'Veuillez saisir les dates de votre absence'. Below this, there are two date selection fields. The first field is labeled 'du' and shows the date '27/09/2023' with a calendar icon to its right. The second field is labeled 'au' and shows the date '04/10/2023' with a calendar icon to its right. Below the second date field, the word 'optionnelle' is written in small text. At the bottom of the form, there is a large blue button with the text 'Envoyer mon absence'. The bottom of the screen shows a standard Android navigation bar with back, home, and recent apps icons.

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MPC - ENG

Make remote monitoring easier using the information and links provided on this page.

HEART FAILURE

[Complete User Guide](#)

[Enter weight and blood pressure manually](#)

[Technical support](#)

[Kit return process](#)

[Terms and Conditions](#)

Complete user guide

COMPLETE USER GUIDE

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Summary

01. My equipment	07. Lab results entry
02. User guide	08. Data visualization
03. Intended use	09. History logs
04. Indications	10. Notify of an absence
05. Precautions	11. My healthcare team
06. Measurements	12. Settings

01. MY EQUIPMENT:

- Tablet and charger
- A blood pressure monitor consisting of a main unit and a cuff
- A scale
- A set of batteries
- A quick user guide
- The packaging box and its insert should be kept for occasional transport or when returning the equipment.

You will find a set of batteries in the kit, in case the ones currently installed run out.

Any additional batteries required will be at your own expense (AAA batteries).

The cuff included in the kit fits an adjustable arm circumference of 22–42 cm. If you require a cuff for an arm circumference of 42–48 cm, please request one from Newcard.

**We kindly ask you to look after this equipment, which is provided for the duration of your monitoring,
and we thank you for taking the utmost care of it.**

The tablet is strictly for personal use and must only be used for the "Newcard" application.

Any other use could impair its proper functioning.

02. USER GUIDE :

The "Newcard" application is a telemonitoring device for chronic heart failure, allowing you to regularly send your measurements such as weight and lab results to your healthcare professionals. The purpose of this system is the potential early detection of cardiac decompensation.

NEWCARD makes the Application available to Patients pursuant to the Decree of June 22, 2023, regarding the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No. 2021-1754 of December 23, 2021, on the financing of social security for 2022.

 This application is a medical device accessory within the meaning of Regulation (EU) 2017/745, class I.

Year of publication of the latest version of the user guide: 2026

Instruction leaflet reference and version: MPC-NTU-PAT-ENO-01

If needed, a printed version can be provided upon request at the following address:
<https://newcard.io/contacts/> or via email at **cardio@newcard.io**

Unique Device Identifier

Legal manufacturer's name
and address

Year of manufacture

Consult instructions for use

CE marking

Catalogue number

Batch code / Lot number

Do not dispose of packaging
and items in household waste

Fragile, handle with care

Keep away from heat

03. INTENDED USE :

"1 Minute For My Heart" is a software as a medical device intended for the telemonitoring of patients suffering from chronic heart failure through the collection, analysis, storage, and visualization of health data. The patient application enables the collection of your data for processing by the Newcard algorithm.

Eligibility criteria:

Patients suffering from chronic heart failure, regardless of the type and etiology of the disease, who meet one of the following two conditions:

1. Hospitalization within the past 12 months for a heart failure flare-up;
2. OR currently in NYHA class 2 or higher.

04. INDICATIONS :

Blood pressure and weight measurement are daily by default.

The frequency of your measurements can be adapted by your cardiologist if needed.

It should ideally be taken at rest, to ensure a reliable transcription of the data.

The connected tablet must be placed on a table and the scale on a flat, hard floor (no carpet or rugs). When taking your weight measurement, please make sure to set your tablet down before stepping onto the scale, so as not to distort the accuracy of your data.

Also ensure you have a chair to sit on while taking your blood pressure.

Technical specifications / prerequisites :

An internet connection is required to use the application. The tablet provided by Newcard is equipped with a 4G SIM card, providing internet access.

Mobile operating system	Compatible version
iOS	≥ 10
Android	≥ 10
Hardware	
Tablets	10-inch screen size with 800 x 1280 resolution
Smartphones	10-inch screen size with 800 x 1280 resolution
Smartphones	• 6.5-inch screen size with 412 x 915 resolution: for Android • 6.5-inch screen size with 414 x 896 resolution: for iPhone.

05. PRECAUTIONS :

This device is for personal use only.

The Newcard service is not an emergency service; it is not intended to replace interventions by healthcare professionals. If you have any medical questions or if your condition suddenly worsens, please contact a healthcare professional or call your local emergency services.

THE KIT IS A LOAN GRANTED TO YOU BY NEWCARD FOR THE DURATION OF YOUR TELEMONITORING. AT THE END OF YOUR MONITORING, YOU ARE REQUIRED TO RETURN THE DEVICE PROVIDED TO YOU AS SOON AS POSSIBLE AND IN THE SAME CONDITION IN WHICH YOU RECEIVED IT.

Preventive and routine maintenance operations:

Newcard may modify or adapt the NEWCARD application or the General Terms and Conditions of Use. In this case, an update on your part may be required. In accordance with the Terms of Use, this update must be carried out promptly.

Residual risks

The software has no direct physical contact with the patient and poses no mechanical, electrical or biological risk. The residual risks identified are of a technical or software nature; although rare, they may, in certain situations, have an indirect impact on patient care. The main residual risks associated with the use of this software are described below.

1. Appropriate technical and organisational measures have been put in place to ensure the platform remains available. Despite these controls, there remains a residual risk that the platform may be temporarily unavailable until it is restored (in the event of an internet connection failure, a technical error, etc.).
2. The platform incorporates mechanisms designed to ensure the reliability of the alerts it generates. Nevertheless, a residual risk of alert errors—including, in particular, false positives, false negatives, duplicates or alerts not being displayed—cannot be entirely ruled out.
3. Despite the controls in place, errors may occur in the transmission of patient measurements (failure to transmit data, or data that is inaccurate or unreliable).
4. Newcard has implemented appropriate cybersecurity measures to ensure the security of all data transmitted. However, a residual risk of loss, alteration or unauthorised access to this data cannot be entirely ruled out. In any event, Newcard is committed to safeguarding users' privacy and complying with applicable data protection regulations, in particular the GDPR.

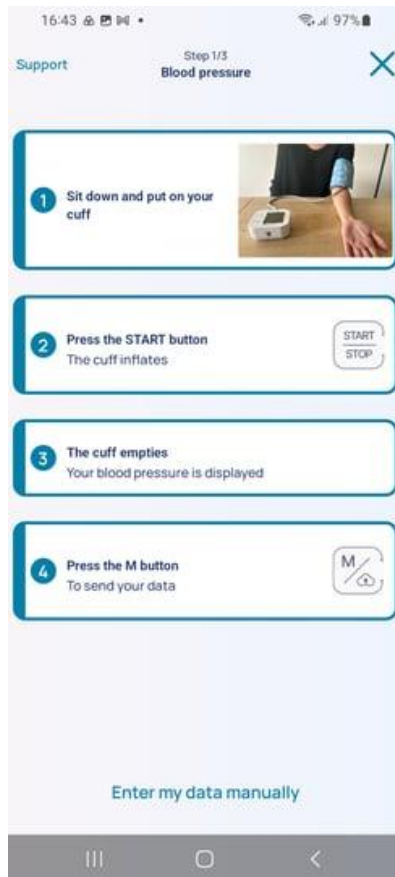
06. MEASUREMENTS :

1. Turn on your tablet by pressing the small button on the right.
2. Tap the NEWCARD icon on the screen.
3. Tap "Take my measurements". The date of your next measurement is shown as an indication.


4. Please follow the instructions displayed on your tablet and click on "**Continue demo**".
5. Sit down and relax. Place the cuff around your left arm and adjust it.
The red band on the cuff must be positioned over the inside of the elbow crease.
6. Place your left arm on a table with the palm of your hand facing upward.


7. Press the "Start / Stop" button on the blood pressure monitor.
The cuff will inflate and then deflate. When the cuff is completely deflated, press the "M/" button.

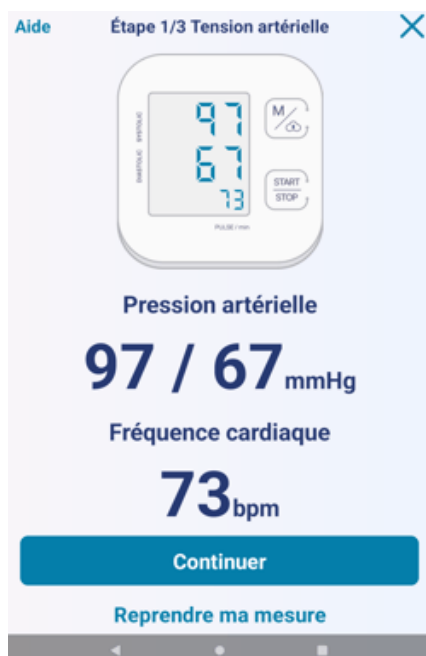8. You can still enter your blood pressure and heart rate manually by tapping "**Enter my data manually**". Your blood pressure and heart rate will be displayed on the tablet. Tap "**Continue**"

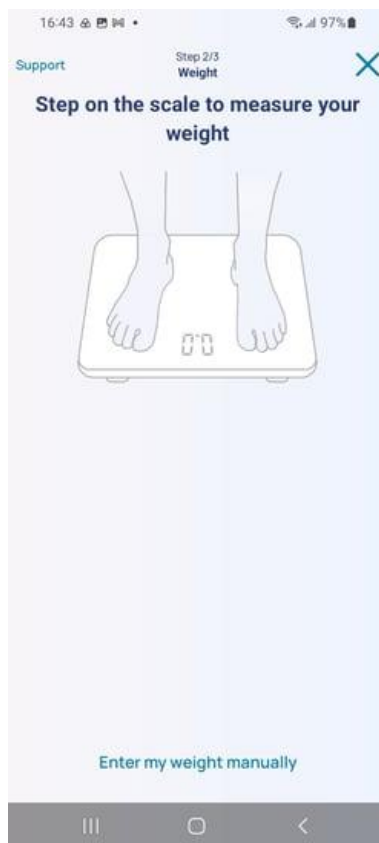9. Step onto your scale.

10. When the measurement is displayed, tap "**Continue**".

You can always manually enter your weight by selecting "**Enter my data manually**".

11. Answer the question: **I feel the same as usual / I don't feel as well as yesterday.**

12. If you feel worse than yesterday, select the symptom(s) or discomfort(s) you are experiencing.

16:43 97%

< Back Step 3/3 Symptoms X

Compared to yesterday, what new discomfort do you feel?

1 or more possible answers

- ☐ I cough more
- ☐ I'm more out of breath
- ☐ I add a pillow to sleep
- ☐ I'm more tired
- ☐ My ankles are more swollen
- Nothing to report

III

15:10 19%

< Back Step 3/3 Symptoms X

Compared to yesterday, what new discomfort do you feel?

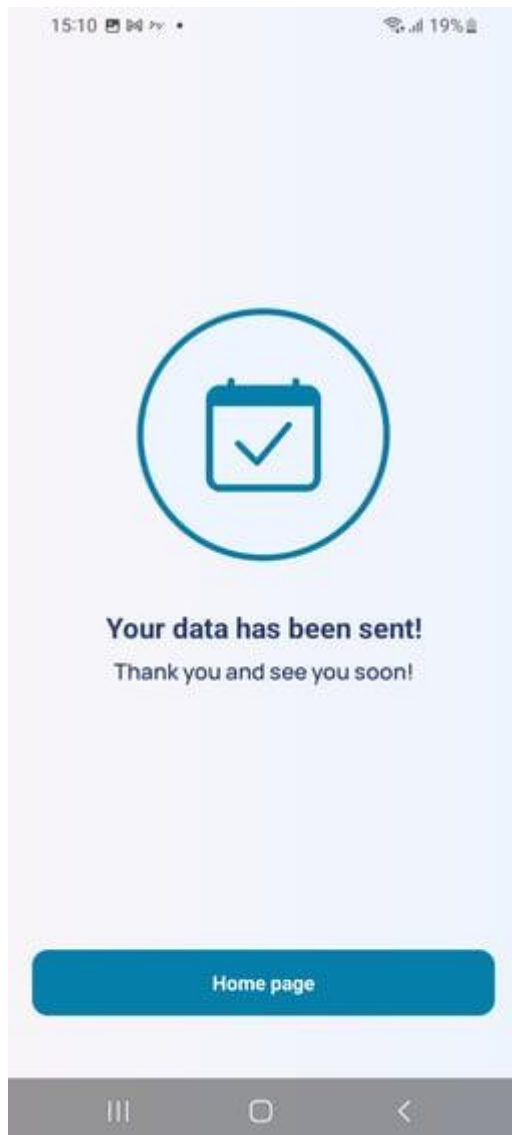
1 or more possible answers

- ☐ I cough more
- ☒ I'm more out of breath
- ☐ I add a pillow to sleep
- ☒ I'm more tired
- ☐ My ankles are more swollen
- Nothing to report

Continue

III

13. Tap **"Continue"**. Your data has been successfully sent. Select **"Home screen"**.

14. Turn off the tablet: press the power button for three seconds. A pop-up will appear: select **"Shut down"**.

07. LAB RESULTS ENTRY :

When your cardiologist prescribes lab tests, you can enter your results into your tablet as soon as you receive them. You can only enter one lab test report per day. In the application, you can enter the date of your test and scroll through the different tabs to fill in the values shown on your lab report.

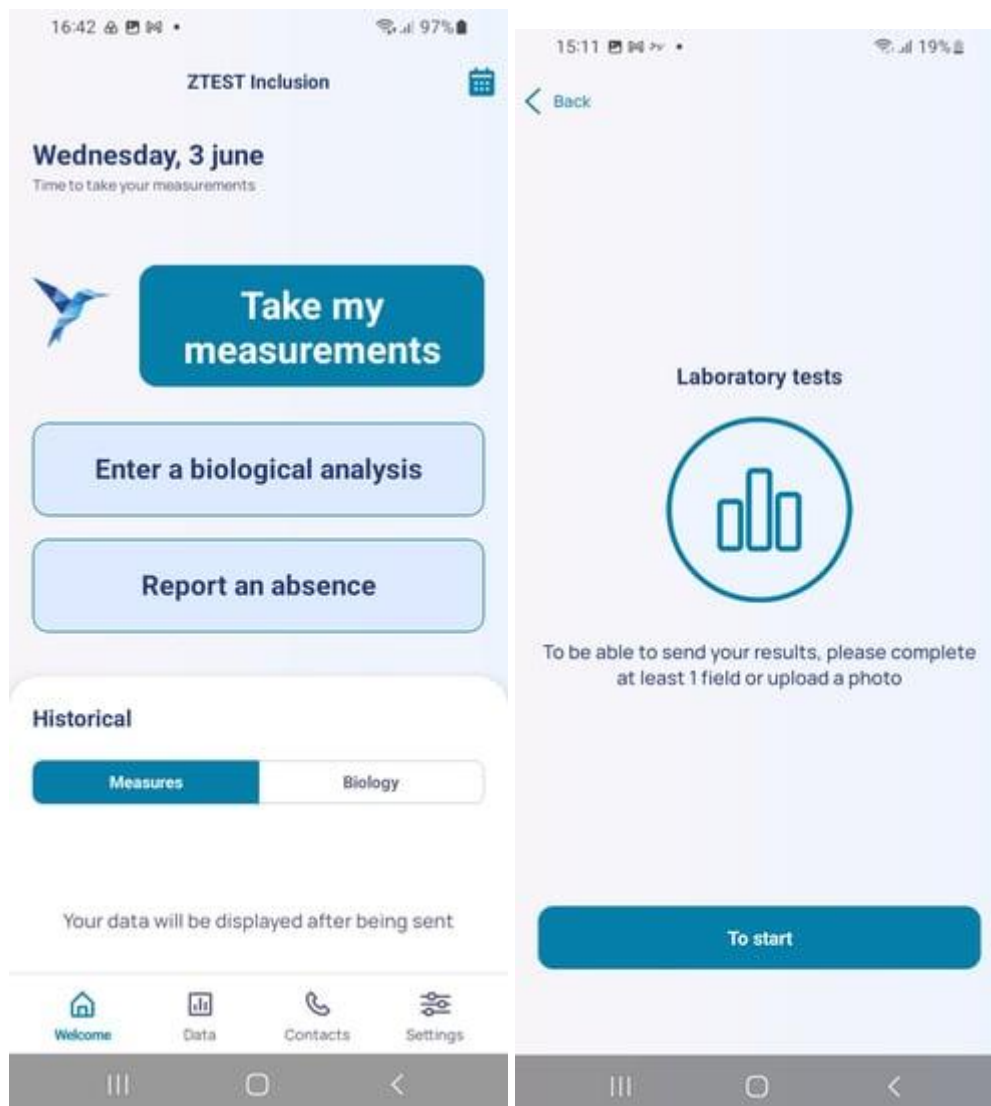
Lab tests are divided into tabs:

Biochemistry, Cardiology parameters, Hematology, and a photo of the report to make entering your results easier.

Your lab data will appear directly on your cardiologist's platform, where they can review it and

access a dated history of your results.

1. Upon receipt of your results, you can enter them into the NEWCARD application.
Tap: "+ **Lab tests**" then "**Start**".

2. Set the date of your analysis report. To do this, select the date directly with the key or click on the calendar button to open the calendar. After selecting the correct date, tap "**Continue**".

The screenshot shows the 'Step 1/4 Biochemistry' screen. At the top, there is a 'Back' button and a close button (X). Below the title, the 'Analysis date' is set to '23/10/2025' with a calendar icon to its right. The form contains several input fields: 'BNP' with the value '52' and unit 'pg/mL', 'NTproBNP' with unit 'pg/mL', 'Creatinine levels' with unit 'mg/L', and 'GFR (CKD-EPI)' with unit 'ml/min/1.73 m²'. At the bottom, there is a numeric keypad with buttons for digits 1-9, 0, a decimal point, and an 'OK' button. There are also icons for voice search, settings, and a magnifying glass.

The screenshot shows the 'Step 2/4 Haematology' screen. At the top, there is a 'Back' button and a close button (X). The section is titled 'Haemogram (blood count)'. It includes input fields for 'Erythrocytes or red blood cells' with unit 'T/L', 'Haemoglobin' with unit 'g/dL', and 'Haemogram (white blood cell count)'. Below this, there is a 'Unit' selector with two options: '/mm3' (selected) and 'G/L'. Further down are input fields for 'Leukocytes or white blood cells' with unit '/mm3', 'Neutrophils (PNN)' with unit '/mm3', 'Eosinophilic polymorphonuclear cells (PNEO)' with unit '/mm3', and 'Basophilic polymorphocytes (BNP)'. At the bottom, there is a large blue 'Continue' button.

3. In the application, lab tests are divided into 4 steps to make data entry easier. You will find the following sections: "**Biochemistry**", "**Cardiology parameters**", "**Hematology**", "**Photos**", and "**Send**". To move to the next section, tap "**Continue**".

4. Each tab contains a list of fields you can complete. To scroll through the list, touch the screen and swipe up.

Fill in the data for which you have information on your lab report. Tap the field you want to complete. The numeric keypad will appear: enter your data by tapping the corresponding numbers. When you have entered all the data you have for a category, tap "**Continue**" to move to the next section.

16:44 97%

Step 3/4
Other

Haematuria

Or enter the value

_____ hematies/mm3 v

Proteinuria over 24 hours

_____ mg/24h

Proteinuria on sample

_____ mg/g

Calcium

_____ mg/L

5. Tap **"Add a photo"**, then **"Take a photo"**. Allow the application to access the camera. Position your results and tap **"Take a photo"** to capture the image. Once the photo is taken, confirm it by tapping **"Continue"**. You can add more photos by tapping the **"+"**. If you are not satisfied with the photo, tap **"Retake photo"** to try again.


15:12 19%

Step 4/4
Photo

Please add photos of each page of your laboratory tests results


Add a photo

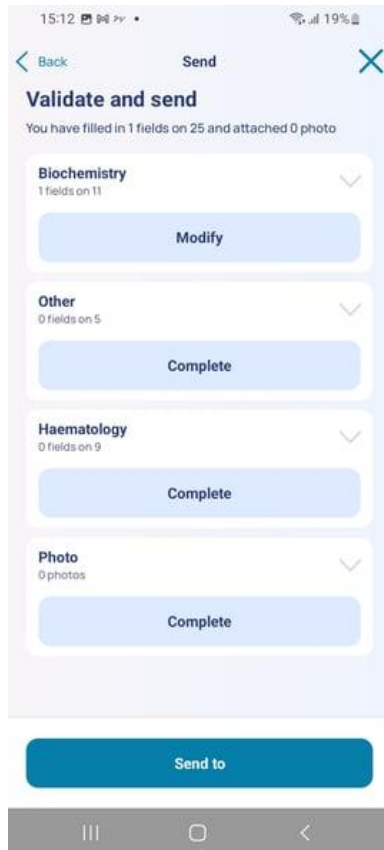6. Once the data is entered and the photo is uploaded, you can still edit the fields. If everything looks correct, tap "**Send to**".

7. Your data has been sent. Select "**Home screen**".

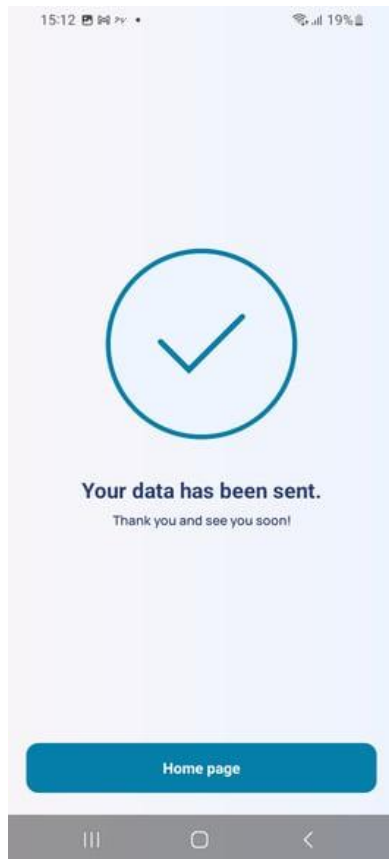Turn off the tablet: press and hold the power button for three seconds. A window will open: tap "**Power off**".

08. VIEWING STATISTICS :

You can access your clinical data history in the form of tables and graphs.
If you want to view your data, tap "**Data**" at the bottom of the home screen.

You can access your weight charts.

To return to the start, tap "**Home**".

Turn off the tablet: press and hold the power button for three seconds. A window will open: tap "**Power off**".

09. HISTORY LOGS:

You can also view your measurement and lab test history by scrolling down on the home screen. In the "**Measurements**" tab, you will find the history of your weight, blood pressure, or heart rate.

In the "**Lab tests**" tab, you can access your lab report history. Select the date you want and tap the arrow to open the lab report you entered.

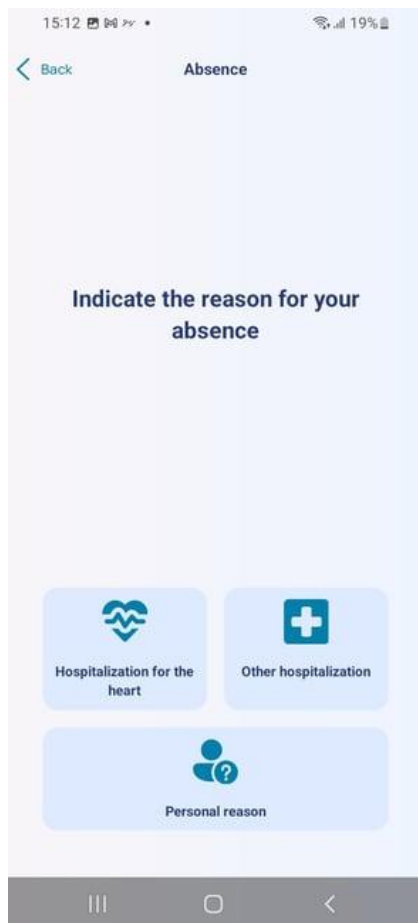
To return to the start, tap "**Home**". Turn off the tablet: press and hold the power button for three seconds. A window will open: tap "**Power off**".

10. NOTIFY OF AN ABSENCE :

1. On the home screen, tap **"Report an absence"**.

2. Select the reason for your absence by tapping **"Personal reason"**, **"Hospitalization for heart condition"**, or **"Other hospitalization"**.

3. Specify your absence by selecting your departure and return dates, then tap "**Send my absence**".

4. Your absence has been successfully sent. Return to the home screen by tapping "**Home screen**".

11. MY HEALTHCARE TEAM :

In the "**Contacts**" tab on the home screen, you will find the Newcard support number and any other useful contacts. To return to the start, tap "**Home**".

12. SETTINGS :

In the "**Settings**" tab, you will find your connected devices, your phone number, your social security number, the option to change your password and use a PIN code, the Legal Terms of Use of the application, and the regulatory information regarding the device (the electronic label) and its version.

Manufacturer's address :
Newcard SAS
Parc du Moulin
236 avenue Clément Ader
59118 Wambrechies

Enter weight and blood pressure
manually

Manually enter weight and blood pressure

This entry allows you to log your data manually in the event that one or all of your devices fail to connect to the app.

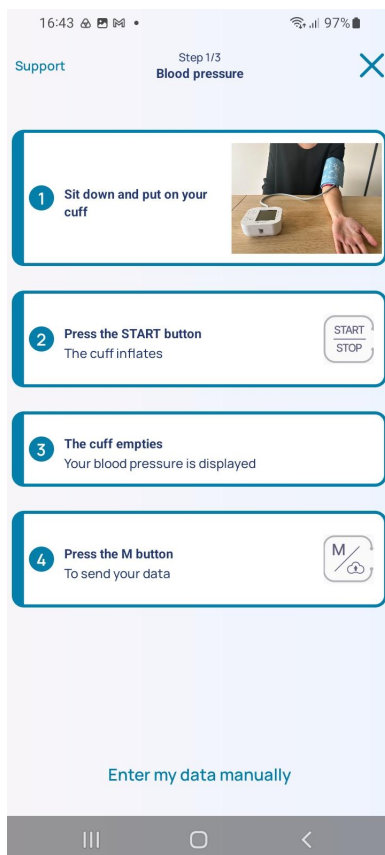
1. Select

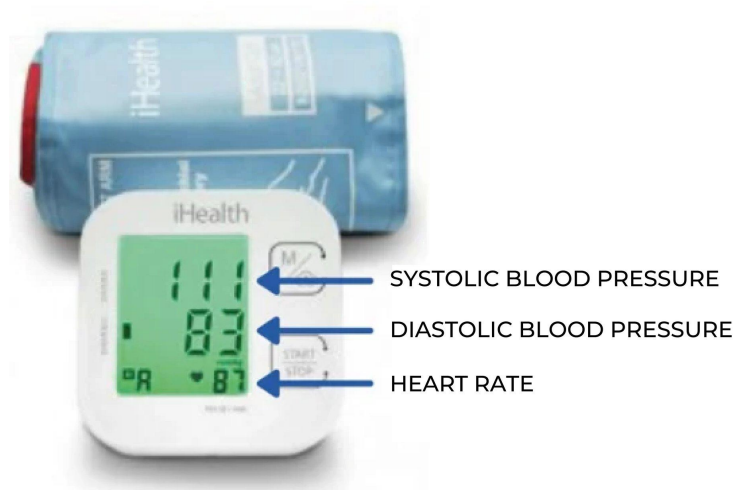
Start the usual measurement procedure.

**Take my
measurements**

2. To take your blood pressure, please follow the instructions displayed on the tablet.

If your data does not appear on the tablet, select: '**Enter my data manually**'.

3. Once the blood pressure monitor has deflated and the readings are stable, remove the cuff and enter your measurements in the appropriate fields.

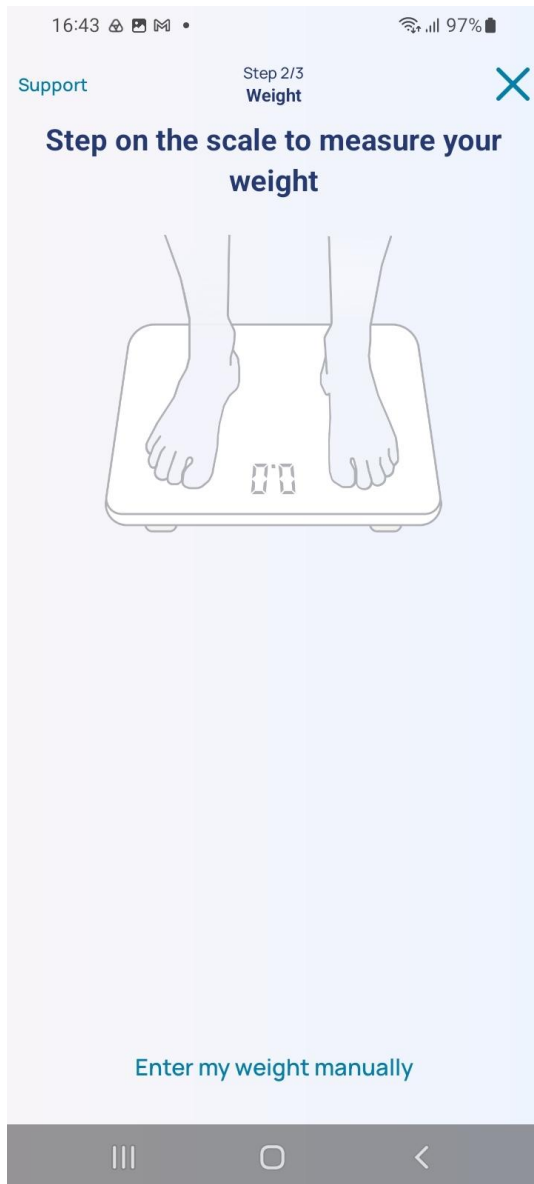
Enter your blood pressure: systolic pressure / diastolic pressure

Select 'Continue' and enter your heart rate.

5. If the error message "We were unable to connect to the scale"

Please try again in 5 seconds " is displayed, or if your weight is not displayed on the tablet, select :

"Enter my weight manually"

6. Enter your weight using the keyboard that appears.

7. Tap "Continue" to proceed to the next screen.

8. Answer the questions

9. Send your data

10. Turn off the tablet: press and hold the power button for three seconds. A window will open: tap "Power off".

Or close the mobile app.

Technical support

Technical support

For any problem you encounter, please start by testing the various options listed below. Follow the solutions with the help of the symbol glossary.

Technical FAQ

Tablet or mobile app issues

The tablet will not turn on

Plug the tablet into the mains for several hours and turn it back on by pressing and holding the power button for **3 seconds**.

The following message appears: "Please allow the use of Bluetooth"

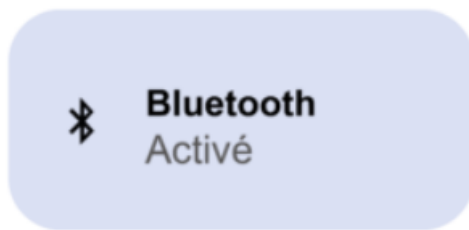
Bluetooth:


Tap "**OK**" to automatically activate Bluetooth, then tap "**Enable**".

You can verify that the Bluetooth icon is activated by swiping down from the top of the tablet screen.

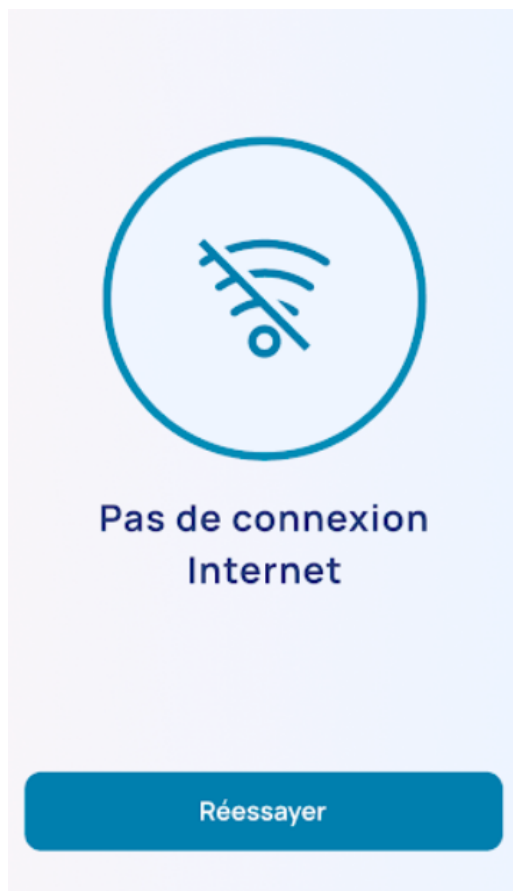The error message appears: **"No internet connection."**

Turn off your tablet by pressing and holding the power button for 3 seconds.

A window will open: Tap **"Turn off"**.

Wait until the tablet is completely turned off.

Turn the tablet back on: press and hold the power button for 3 seconds.

Repeat the measurement procedure.

If the message is still displayed, please contact the Newcard support team.

The following message appears: "Turn on your location in settings"

Tap "**Turn on location**".

And enable the use of Location services in the settings. You can verify that the Location icon is activated by swiping down from the top of the tablet screen.

The following message appears: "Error loading data. Please try again"

Tap "**Reload**".

If it still does not work, restart the tablet: turn off your tablet by pressing and holding the power button for 3 seconds.

A window will open: Tap "**Turn off**".

Wait until the tablet is completely turned off.

Turn the tablet back on: press and hold the power button for 3 seconds.

If the problem persists, please contact the Newcard support team.

The tablet goes into "Reboot" or "Safe" mode.

Press and hold the power button **for 8 seconds** until the tablet restarts.

Issue with the scale

The scale displays the word "LO"

Turn the scale over and change the batteries (AAA batteries).

Repeat the measurement procedure as usual.

When measuring weight, an error message appears: "The application has stopped working."

Changing the scale batteries:

Turn the scale over and change the batteries (AAA batteries).

Repeat the measurement procedure as usual.

Blood pressure monitor issue

Blood pressure monitor will not turn on.

Turn the blood pressure monitor over and replace the batteries (AAA batteries).

Take the measurement again, as usual.

The blood pressure monitor displays 'Er 0'.

Take your blood pressure reading again while remaining still.

The blood pressure monitor displays 'Er 4'.

Cuff position:

Reposition the cuff, ensuring it is fastened more tightly.

Cuff-to-monitor connection:

Check that the cuff tube is firmly pushed into the blood pressure monitor.

If the problem persists, contact Newcard to request a replacement cuff.

When taking a blood pressure measurement, an error message appears: 'The application has stopped working'.

Replacing the blood pressure monitor batteries:

Turn the blood pressure monitor over and replace the batteries (AAA batteries). Take the measurement again, as usual.

If the problem persists, please contact Newcard.

For any other technical questions not mentioned above, please contact us.

TERMS AND CONDITIONS

TERMS AND CONDITIONS

LEGAL NOTICE FOR THE 'NEWCARD' APPLICATION

Patients are kindly requested to read the following Terms of Service for the 'Newcard' application carefully. We recommend that users refer back to these legal notices regularly, as they are subject to change.

ARTICLE 1 – GENERAL TERMS OF USE"

The purpose of the 'Newcard' application is to collect measurements taken by patients at home and transmit them via 3G or Wi-Fi to the platform developed by Newcard, which is accessible to healthcare professionals involved in telemonitoring. Its use is strictly reserved for patients suffering from heart failure who have been enrolled in the program by a physician and on whose device the application has been installed.

Particularly for the purpose of improvement, Newcard may modify or adapt the application. In such cases, an update by the patient may be required. The patient is required to perform this update as soon as possible.

Newcard also reserves the right to modify the application's terms of use at any time (by adding, changing, or removing elements). In this case as well, an update to the application by the patient may be necessary. Following each update, users are advised to review the legal notice.

ARTICLE 2 – INTELLECTUAL PROPERTY

This 'Newcard' application is published by Newcard, located at 236 avenue Clément Ader, 59118 Wambrechies, and is hosted by ATE (Avenir Télématique), 21 Avenue de la Créativité, 59650 Villeneuve-d'Ascq.

The elements created for this application are the exclusive property of Newcard or, where applicable, of third parties that have authorized their citation by the company. All elements (including trademarks, logos, images, graphics, icons, audio, software, and domain names) displayed on the 'Newcard' application are protected by applicable intellectual property laws and belong to Newcard or are subject to an authorization for use.

The user is granted only a right to use the information displayed (texts, logos, designs, images, etc.) on the application. This right is limited to personal and private use; the user has no right of reproduction, representation, modification, publication, or adaptation of all or part of the application's elements, except for a single copy intended strictly for private use, and subject to compliance with intellectual property rights and any other property rights mentioned. Any other use is subject to the prior agreement of Newcard. Consequently, any unauthorized use will constitute infringement, for which Newcard reserves the right to pursue legal remedies.

ARTICLE 3 – CONTACT INFORMATION

The owner and publisher of the 'Newcard' application is the company NEWCARD.

Newcard

236 Avenue Clément Ader

59118 Wambrechies,

France contact@newcard.io +33 (0)3 20 14 71 14

Registered with the Lille Métropole Trade and Companies Register (RCS) under number 821 766 961

SAS with a share capital of 3,511.00 euros

Managing Director: Pierre Emmanuel BONNET

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59650 Villeneuve d'Ascq, France

SAS with a share capital of 60,000 €

Lille Trade and Companies Register (RCS) No. 347 607 764

www.ate.info

+33 (0)3 28 800 300

Managing Director: Maxence ROUSSEAU

Questions or requests for information regarding the application may be sent to the company via the aforementioned postal or email addresses, or by phone using the contact details provided.

ARTICLE 4 – APPLICABLE LAW

The 'Newcard' application is an application created in France. Any dispute arising in connection with its use shall be governed by French law, and the French courts shall have exclusive jurisdiction.

For any questions regarding personal data or liability, users are invited to read the application's Terms of Service, which they have accepted.

TERMS OF SERVICE FOR THE 'NEWCARD' APPLICATION Please find attached the Terms of Service that you accept / accepted during your first log-in to the application. For any withdrawal, please contact NEWCARD using the contact details provided.

Between NEWCARD, a Simplified Joint-Stock Company (SAS) registered under identification number 821 766 961 with the Lille Métropole Trade and Companies Register, having its registered office located at 391 avenue Clément Ader, 59118 Wambrechies, represented by its legal representative (hereinafter referred to as the 'Service Provider' or 'NEWCARD')

And the patient, defined herein as a user of the Telemonitoring Software prescribed by their physician in accordance with the legal and regulatory requirements of the Social Security

(hereinafter referred to as the 'Patient'). NEWCARD and the Patient are hereinafter collectively referred to as the 'Parties' or individually as a 'Party'.

1. ACCEPTANCE AND UPDATES

Important : Any use of the Logiciel in any capacity whatsoever implies the unreserved acceptance by the end user (the Patient) of these TOU.

1.1 The Patient acknowledges that he/she has read, understood and accepted these TOU without limitation or condition, initially by giving his/her free and unreserved consent to his/her physician to be tele-surveilled and use the "Newcard" Application, then by accepting these TOU on the Application (i.e. the "System" or the "Software"). These TOU constitute the complete and exclusive agreement between the Patient and NEWCARD concerning the use of the Logiciel.

1.2 NEWCARD reserves the right to change, modify all or part of these TOU. The Patient is informed of these modifications and of the date of updating of the TOU.

If the Patient does not accept these GTU or any updated version of these GTU, he must not use the System.

1.3 The Patient hereby warrants and confirms that he/she is of legal age to consent to these TOU and that he/she is fully capable of (i) concluding the contract that these TOU constitute, and (ii) complying with them.

2. PURPOSE

2.1 NEWCARD makes the Application available to Patients under the Decree of June 22, 2023, concerning the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No. 2021-1754 of December 23, 2021, on social security financing for 2022.

2.2 NEWCARD makes the Application available to the Patient and grants them a non-exclusive, personal, and non-transferable license to use the System solely within the French territory.

2.3 The System offers the following main features:

- Weight tracking using a connected scale;
- Blood pressure and heart rate measurement using a connected blood pressure monitor;
- Symptom reporting through a questionnaire consisting of five (5) questions;
- Information logging for biological data;

- Application on a digital tablet or smartphone allowing the retrieval and visualization of measurements taken, as well as notification of absence;
- A 3G system for sending measurements, particularly for patients who do not have an internet connection;
- An algorithm allowing data processing and the triggering of alerts in the event of a variance;
- A telephone service to answer technical questions, on business days from 9:00 AM to 5:00 PM.

3. FREE SERVICE AND CONNECTION CONDITIONS

3.1 The provision and use of the System are free of charge for eligible patients under the Decree of June 22, 2023, concerning the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No. 2021-1754 of December 23, 2021, on social security financing for 2022, and to whom the telemonitoring service has been medically prescribed.

3.2 The Patient acknowledges that they possess the necessary means and skills to use the System. Consequently, the Patient accepts and understands that the performance of the System may be affected by the means they choose to use. Furthermore, if the Patient uses their own internet connection to send their data, they are responsible for their internet subscription, and NEWCARD shall not be held liable for the billing terms of any third-party operator.

3.3 In accordance with Article L.221-28 of the French Consumer Code, the Patient does not benefit from a right of withdrawal once they validate their registration to the System, thereby accessing the System's digital content. In any event, the Patient may withdraw their consent at any time by contacting NEWCARD using the contact details provided to request the closure of their account on the System.

4. OBLIGATIONS OF NEWCARD

4.1 The Service Provider makes the System available at the Patient's home and offers telephone support in the event of a technical issue, available from Monday to Friday from 9:00 AM to 5:00 PM.

4.2 The Service Provider undertakes to host the Patient's data within the System through a hosting provider certified to store health data (HDS certified hosting provider).

4.3 The Service Provider undertakes to comply with the provisions of Regulation (EU) 2016/679 on the protection of natural persons with regard to the processing of personal data and on the free movement of such data, and repealing Directive 95/46/EC (General Data Protection Regulation).

4.4 The Service Provider undertakes, within the framework defined by these Terms of Service, to use all reasonable means to ensure the performance and continuity of access to the System. However, the Patient accepts and acknowledges that maintenance and update operations on the System may occur, and the Service Provider will use its best efforts to notify the Patient in advance.

4.5 The Service Provider undertakes to comply with the applicable regulatory provisions relating to its status as a regulated Medical Device and, as such, bears the CE marking.

5. MEDICAL DISCLAIMER

5.1 The Software provides and publishes informative content based on the results of the measurements taken by the Patient and is solely intended to assist them and provide information required by the System for the proper conduct of telemonitoring. NEWCARD is not a medical organization or a healthcare professional. NEWCARD offers a System for transmitting and organizing information sent by the Patient. In this regard, the System informs physicians of the data sent. None of the content on the Software shall be interpreted to mean otherwise. Consequently, the information accessed on the Software shall not be considered a substitute for (a) a personal and professional medical consultation provided by a physician, and/or (b) a diagnosis, interpretation, or evaluation made by the physician either remotely or in person.

5.2 THIS SYSTEM IS NOT AN EMERGENCY SYSTEM. Its purpose is the daily monitoring of data sent in order to notify the physician in the event of a significant variance in the data. Telemonitoring follow-up does not exempt the Patient from their usual care routines. In the event of any medical difficulty or emergency, entirely left to their own judgment, the Patient must contact healthcare professionals as they normally would (for example, their general practitioner or emergency medical services).

5.3 NEWCARD shall not be held liable:

- If data fails to reach it, for any reason whatsoever, or if the data received is illegible or impossible to process;
- If the user is unable to access or use the application, for any reason whatsoever;
- If, for any reason whatsoever, the connection is interrupted.

6. PATIENT OBLIGATIONS

6.1 The Patient undertakes to allow NEWCARD to create an account for them and to train them on how to use the System if necessary. They then undertake to take their measurements regularly and in accordance with the recommendations made by Newcard for their telemonitoring follow-up. This is an essential condition for the proper functioning of the telemonitoring system.

6.2 The Patient, as a user, acknowledges that they possess the necessary means and skills to use the Software. Consequently, the Patient accepts and understands that the performance of the Software may be affected by the chosen connection means.

6.3 The Patient undertakes to make strictly personal use of the equipment provided to them, in accordance with its intended purpose. They shall follow the instructions provided during installation and/or the information included in the user guide. They are aware that they have custody and responsibility of the equipment, and notably the tablet equipped with the Application. They are also aware that the trend curves of their data can be visualized when connected to the internet or a 3G server. In this respect, they are solely responsible for any use of the tablet and the Application by third parties, and must therefore take the necessary security measures. On their own initiative, they may notably lock the tablet or smartphone on which they use the Application with a confidential code of their choice. In the event that the tablet provided by Newcard is locked by a confidential code, they must imperatively remove this code before returning the equipment to Newcard.

6.4 The Patient undertakes to use their best efforts to enter their data into the System in a reliable, complete, accurate manner, and in accordance with the formats agreed upon between the Parties. In case of doubt or difficulty, the Patient may contact Newcard for assistance with data entry.

6.5 The Patient undertakes to comply with the technical prerequisites and hardware configurations recommended by the Service Provider.

6.6 At the end of their telemonitoring follow-up or upon termination of the Contract by either Party, for any reason whatsoever, the Patient or their beneficiaries (ayants droit) undertake to return the equipment to NEWCARD within the following 30 days, by sending it to the address provided to them. The Patient also undertakes to return the equipment to NEWCARD in the event of defective equipment or any other reason requiring a kit replacement.

6.7 The Patient undertakes to comply with the provisions of these Terms of Service and the requirements of the guidelines set by the Social Security.

6.8 The Patient undertakes to comply with applicable laws and regulations in force.

7. COMMON OBLIGATIONS

7.1 Each Party undertakes to inform the other Party of any event or information brought to its knowledge that would be likely to have any effect whatsoever on the Software and these Terms of Service.

7.2 Each Party undertakes to respect the confidentiality of these presents and not to harm the image and reputation of the other Party.

8. INTELLECTUAL PROPERTY

8.1 NEWCARD remains the sole owner of all its intellectual and material property rights regarding trademarks, photographs, computer programs, distinctive signs, know-how, methods, and all documents used or implemented during the use of the Software, and notably its algorithm (all such rights and elements hereinafter referred to as the "Elements").

8.2 The Patient undertakes not to infringe upon, or challenge in any manner whatsoever, the Service Provider's rights over the Elements belonging to it.

8.3 The Contract does not imply any transfer of ownership over the Software, and notably over the data, methods, or development tools for which the Service Provider holds proprietary or licensing rights and which may have been made available to the Patient for the purposes hereof.

8.4 The Patient undertakes not to make any modification to the Software. The Patient acknowledges that the Software is protected by intellectual property rights and declares that they have not entered into this contract with the intent to infringe upon them in any manner whatsoever. In particular, the Patient undertakes not to copy it, disclose it in any way whatsoever, and not to commit or allow any action to be committed that is likely to infringe upon these rights.

9. LICENSE FOR USE

9.1 The Service Provider hereby grants to the Patient, who accepts it, a personal, non-transferable, non-assignable, and non-exclusive license to use the Software for a term of 2 years, tacitly renewable. This license shall terminate at the end of their telemonitoring follow-up.

9.2 The granting of the right of use, as provided for in the Contract, is consented solely to the Patient. This grant is validly attributed solely for the French territory from the moment a session is opened on the Software.

9.3 The Patient is prohibited from any type of exploitation not explicitly authorized by these Terms of Service, and notably from:

- Making a backup copy for any reason other than preserving the use of the Software;
- Correcting or having a third party correct any potential errors or bugs in the Software without the prior written consent of the Service Provider;
- Granting a loan or making the Software or its documentation available by any means whatsoever, including via the Internet;
- Adapting, arranging, or modifying the Software.

10. ENTRY INTO FORCE / TERM

10.1 This document enters into force as of the date of acceptance of these Terms of Service.

10.2 The duration of the Contract is 2 years.

10.3 The Contract shall be tacitly renewed for successive periods of two years, except in the event of (a) termination by either Party notified by registered letter with acknowledgment of receipt subject to one (1) month's notice, or (b) the end of the medical telemonitoring follow-up and non-renewal of the prescription by the Cardiologist.

11. CONFIDENTIALITY

11.1 The term "Confidential Information" hereinafter refers to any information, whether materialized or not, that is strategic, financial, technical, or commercial. This confidential nature may notably result from the presence of any appropriate marking, from the very nature of the document, or from an express oral indication.

11.2 The Parties undertake to ensure that Confidential Information is kept confidential by taking all useful and reasonable security measures, and used solely for the purposes hereof.

11.3 The provisions of this article shall remain valid throughout the term of the Contract and for 4 years thereafter.

12. PERSONAL DATA

12.1 Identity of the Data Controller

The collection and processing of personal data are carried out by the Company Newcard, responsible for the application made available to patients. Newcard is domiciled at the following address: 391 avenue Clément Ader, 59118 Wambrechies. It can be contacted at +33 (0)3 20 14 71 14 or by email at info@newcard.io.

Newcard has appointed a Data Protection Officer (hereinafter "DPO"), duly declared to the French National Commission for Information Technology and Civil Liberties (hereinafter "CNIL"). The DPO can be contacted at +33 (0)3 60 85 06 86 or by email at mes-donnees-personnelles@newcard.io.

12.2 Purposes of Personal Data Processing

The data collected is processed for specific purposes. Consequently, it cannot be used for purposes other than those specified in these Terms of Service. In this case, these purposes are of several types:

- Conducting the telemonitoring of patients suffering from chronic pathologies;
- Providing therapeutic support; and
- Conducting analyses and data studies on patient populations. The objective is to contribute to scientific research on chronic pathologies and their management, telemonitoring, and its medico-economic benefits.

12.3 Legal Basis for Processing

The collection and processing of data are carried out in accordance with the provisions of Regulation (EU) 2016/679 of the European Parliament and of the Council of 27 April 2016 on the protection of natural persons with regard to the processing of personal data and on the

free movement of such data (GDPR), and Law No. 78-17 of January 6, 1978, relating to information technology, files, and civil liberties. The legal basis for processing is the consent of the data subject.

12.4 Categories of Processed Data

Only the data necessary for the patient's telemonitoring within the framework of the tele-follow-up application is required.

This includes the patient's identity data (last name and first name, gender, date of birth), their social security number, their daily vitals (weight, height, blood pressure, heart rate, symptom questionnaire), connection data, and the software usage history.

12.5 Recipients of Personal Data

The collected information is secured and hosted by an HDS certified hosting provider ("Hébergeur de données de santé"). Our data host is ATE, whose contact details are as follows:

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59650 Villeneuve d'Ascq

<https://ate.info> / contact_web@ate.info / +33 (0)3 28 80 03 00

Several persons, whether belonging to Newcard or not, are authorized to access the patient's personal data:

- The cardiologist and their medical team have access to the patient's personal data (sensitive and non-sensitive) in order to ensure their telemonitoring;
- Internal healthcare professionals at Newcard also have access to the data (sensitive and non-sensitive) to ensure the efficiency of the patient's follow-up; and
- Newcard's administrative staff also has access to the patient's non-sensitive personal data (identity, contact details, etc.).

12.6 Retention Period of Personal Data

All data collected and processed by Newcard within the framework of the application is retained:

- For patients followed by a private practice cardiologist: 5 years after the end of their follow-up;
- For patients followed by a healthcare institution (hospital or private clinic): 20 years after the end of their follow-up. At the end of the retention period, the data will be irreversibly anonymized.

12.7 Rights of Data Subjects

Regarding the collection and processing of personal data, data subjects possess numerous rights necessary to guarantee fair and transparent processing. To exercise these rights, they may contact Newcard at the contact details mentioned in paragraph 12.1. Thus, they have a right to:

- **Access to personal data:** The data subject has the right to obtain from Newcard confirmation as to whether or not personal data concerning them is being processed, and, where that is the case, access to said data.
- **Rectification of personal data:** The data subject has the right to obtain from Newcard the rectification of inaccurate and/or incomplete personal data.
- **Erasure of personal data:** The data subject has the right to obtain from Newcard the erasure of personal data concerning them without undue delay.
- **Restriction of processing:** The data subject has the right to obtain from Newcard the restriction of processing of their personal data.
- **Objection to processing:** The data subject has the right to object, at any time, to the processing of personal data concerning them.
- **Data portability:** The data subject has the right to receive the personal data concerning them, which they have provided to Newcard, in a structured, commonly used, and machine-readable format.
- **Withdrawal of consent:** The data subject has the right to withdraw their consent to the processing of their personal data at any time, without affecting the lawfulness of processing based on consent before its withdrawal.
- **Lodging a complaint with a supervisory authority:** The data subject has the right to lodge a complaint with the competent supervisory authority. They may address their remarks to the Commission Nationale de l'Informatique et des Libertés (CNIL), whose contact details are as follows: Commission Nationale de l'Informatique et des Libertés 3 Place de Fontenoy – TSA 8071575334 Paris Cedex 07

12.8 Transfer Outside the European Union

The data collected and processed within the framework of telemonitoring is hosted by ATE – an HDS certified data host – based in Villeneuve d'Ascq.

Our host certifies that the data is hosted within the European Union and that no transfer outside the European Union is carried out. Newcard undertakes to provide any interested person, upon request, with the hosting certificate issued by ATE.

13. TERMINATION - EXPIRATION

13.1 Under the conditions provided for in Article 10.3, the Parties may terminate the Contract. This termination shall take effect 30 calendar days from the receipt of the termination notice.

13.2 The prescription by the Physician of the Newcard service under the Decree of June 22, 2023, concerning the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No.

2021-1754 of December 23, 2021, on social security financing for 2022, is an essential condition of these Terms of Service. In the event of the end of the medical telemonitoring follow-up, the Contract shall terminate automatically, except for authorized out-of-framework follow-ups, which require the express agreement of Newcard.

13.3 In the event of non-performance of their obligation to return the equipment, the Patient will be served a formal notice to pay the sum of two hundred (200) euros to NEWCARD, representing the buyback price of the equipment, by check or bank transfer using the details provided to them by mail.

13.4 Upon expiration or termination of the Contract for any reason whatsoever, the obligations of the Parties shall survive regarding intellectual property and copyright, confidentiality, and those clauses which by their nature must remain in force shall remain in force, such as the "Liability" clause.

14. SUSPENSION – TERMINATION OF ACCESS

14.1 The Service Provider may automatically suspend access to the Software and remove the Software from online availability, without compensation to the Patient and without prior formal notice, in the following cases: (1) upon the Patient's request, (2) due to non-compliance with the Patient's obligations, (3) the Patient's death, (4) discontinuance of the Software, (5) when the Patient's actions endanger the Software in any manner whatsoever, (6) in the event of a material breach of this Contract, (7) in the event of the end of a follow-up semester and non-re-inclusion of the patient by their cardiologist. This decision shall be brought to the Patient's attention by the Service Provider within a reasonable timeframe.

14.2 In the event of suspension of access or termination of the Contract, the Patient has the possibility to extract their data in accordance with applicable regulations.

15. LIABILITY

15.1 The Service Provider shall not be held liable in any manner whatsoever for the consequences resulting from (i) the Patient's lack of diligence in taking their vitals, (iii) the incorrect use of the Software by the Patient, (iv) the use by any third party of the equipment under the sole custody of the Patient, (v) any delay, inaccuracy, or malfunction due to a failure, fault, negligence, or omission of the Patient, their physician, their subcontractors, or any third party over whom the Service Provider has no power of control or supervision.

15.2 Neither Party shall be held liable for the consequences of indirect damages. In particular, data losses are considered indirect damages.

15.3 It is up to the Patient to inform the Service Provider of any defect or non-conformity observed, and to provide proof of the Service Provider's failure.

15.4 The Service Provider shall not be held liable in any manner whatsoever for (v) total or partial unavailability to access the Software, notably for technical maintenance operations or

updating of published information, (vi) viral attacks, (vii) incomplete or erroneous information provided by a user, (viii) the processing or lack of processing of alerts by the Physician.

16. FORCE MAJEURE

Neither Party shall be held liable for any breach of its obligations under the Contract if such breach results from a situation constituting force majeure as defined by the case law of the French Cour de Cassation on the date of its occurrence. Each Party shall immediately inform the other Party of the occurrence of a force majeure event likely to prevent it from executing its obligations. The Parties shall then endeavor to take appropriate measures to mitigate the consequences of the event. However, if the event persists beyond 30 calendar days, the Contract may be terminated by the most diligent Party, without compensation.

17. MISCELLANEOUS

17.1 If any of the clauses of the Contract were to be declared null and void, the remaining provisions of the Contract shall remain in full force and effect.

17.2 The failure of either Party to invoke a breach by the other Party of any of the obligations referred to in the Contract shall not be construed for the future as a waiver of its right to invoke said breach or to enforce the obligation in question.

18. GOVERNING LAW – JURISDICTION

18.1 The Contract is governed by French law.

18.2 UNLESS CONTRARY TO PUBLIC POLICY PROVISIONS, ANY DISPUTE RELATING TO THE APPLICATION, PERFORMANCE, OR INTERPRETATION OF THE CONTRACT SHALL, BY EXPRESS AGREEMENT, BE BROUGHT BEFORE THE COURTS OF LILLE.

MPC - ST - ENG

Complete user guide-

COMPLETE USER GUIDE

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Summary

01. <u>My equipment</u>	07. <u>Lab results entry</u>
02. <u>User guide</u>	08. <u>Data visualization</u>
03. <u>Intended use</u>	09. <u>History logs</u>
04. <u>Indications</u>	10. <u>Notify of an absence</u>
05. <u>Precautions</u>	11. <u>My healthcare team</u>
06. <u>Measurements</u>	12. <u>Settings</u>

01. MY EQUIPMENT:

- Tablet and charger ;
- A scale ;
- A set of batteries ;
- A quick user guide ;
- The packaging box and its insert should be kept for occasional transport or when returning the equipment.

You will find a set of batteries in the kit, in case the ones currently installed run out.

Any additional batteries required will be at your own expense (AAA batteries).

**We kindly ask you to look after this equipment, which is provided for the duration of your monitoring,
and we thank you for taking the utmost care of it.**

The tablet is strictly for personal use and must only be used for the "Newcard" application.

Any other use could impair its proper functioning.

02. USER GUIDE :

The "Newcard" application is a telemonitoring device for chronic heart failure, allowing you to regularly send your measurements such as weight and lab results to your healthcare professionals. The purpose of this system is the potential early detection of cardiac decompensation.

NEWCARD makes the Application available to Patients pursuant to the Decree of June 22, 2023, regarding the registration of medical telemonitoring activities on the list provided for in

Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No. 2021-1754 of December 23, 2021, on the financing of social security for 2022.

 This application is a medical device accessory within the meaning of Regulation (EU) 2017/745, class I.

Year of publication of the latest version of the user guide: 2026

Instruction leaflet reference and version: MPC-NTU-PAT-ENO-01

If needed, a printed version can be provided upon request at the following address:
<https://newcard.io/contacts/> or via email at cardio@newcard.io

Unique Device Identifier

Legal manufacturer's name
and address

Year of manufacture

Consult instructions for use

CE marking

Catalogue number

Batch code / Lot number

Do not dispose of packaging
and items in household waste

Fragile, handle with care

Keep away from heat

03. INTENDED USE:

"1 Minute pour mon Cœur" is a software as a medical device intended for the telemonitoring of patients suffering from chronic heart failure through the collection, analysis, storage, and

visualization of health data. The patient application enables the collection of your data for processing by the Newcard algorithm.

Eligibility criteria:

Patients suffering from chronic heart failure, regardless of the type and etiology of the disease, who meet one of the following two conditions:

1. Hospitalization within the past 12 months for a heart failure flare-up;
2. OR currently in NYHA class 2 or higher.

04. INDICATIONS:

Weight measurement is daily by default.

The frequency of your measurements can be adapted by your cardiologist if needed.

It should ideally be taken at rest, to ensure a reliable transcription of the data.

The connected tablet must be placed on a table and the scale on a flat, hard floor (no carpet or rugs). When taking your weight measurement, please make sure to set your tablet down before stepping onto the scale, so as not to distort the accuracy of your data.

Technical specifications / prerequisites:

An internet connection is required to use the application. The tablet provided by Newcard is equipped with a 4G SIM card, providing internet access.

Mobile operating system	Compatible version
iOS	≥ 10
Android	≥ 10
Hardware	
Tablets	10-inch screen size with 800 x 1280 resolution
Smartphones	10-inch screen size with 800 x 1280 resolution
Smartphones	• 6.5-inch screen size with 412 x 915 resolution: for Android • 6.5-inch screen size with 414 x 896 resolution: for iPhone.

05. PRECAUTIONS:

This device is for personal use only.

The Newcard service is not an emergency service; it is not intended to replace interventions by healthcare professionals. If you have any medical questions or if your condition suddenly worsens, please contact a healthcare professional or call your local emergency services.

THE KIT IS A LOAN GRANTED TO YOU BY NEWCARD FOR THE DURATION OF YOUR TELEMONITORING. AT THE END OF YOUR MONITORING, YOU ARE REQUIRED TO RETURN THE DEVICE PROVIDED TO YOU AS SOON AS POSSIBLE AND IN THE SAME CONDITION IN WHICH YOU RECEIVED IT.

Preventive and routine maintenance operations:

Newcard may modify or adapt the NEWCARD application or the General Terms and Conditions of Use. In this case, an update on your part may be required. In accordance with the Terms of Use, this update must be carried out promptly.

Residual risks

The software has no direct physical contact with the patient and poses no mechanical, electrical or biological risk. The residual risks identified are of a technical or software nature; although rare, they may, in certain situations, have an indirect impact on patient care. The main residual risks associated with the use of this software are described below.

1. Appropriate technical and organisational measures have been put in place to ensure the platform remains available. Despite these controls, there remains a residual risk that the platform may be temporarily unavailable until it is restored (in the event of an internet connection failure, a technical error, etc.).
2. The platform incorporates mechanisms designed to ensure the reliability of the alerts it generates. Nevertheless, a residual risk of alert errors—including, in particular, false positives, false negatives, duplicates or alerts not being displayed—cannot be entirely ruled out.
3. Despite the controls in place, errors may occur in the transmission of patient measurements (failure to transmit data, or data that is inaccurate or unreliable).
4. Newcard has implemented appropriate cybersecurity measures to ensure the security of all data transmitted. However, a residual risk of loss, alteration or unauthorised access to this data cannot be entirely ruled out. In any event, Newcard is committed to safeguarding users' privacy and complying with applicable data protection regulations, in particular the GDPR.

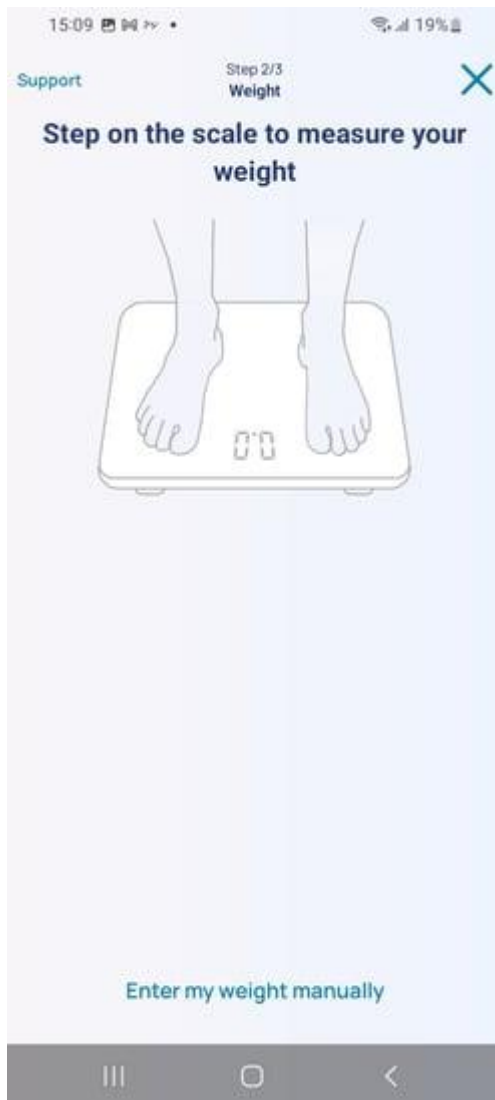
06. MEASUREMENTS:

1. Turn on your tablet by pressing the small button on the right.
2. Tap the NEWCARD icon on the screen.
3. Tap "Take my measurements". The date of your next measurement is shown as an indication.

4. Please follow the instructions displayed on your tablet and click on "**Continue demo**"

5. Step onto your scale.

6. When the measurement is displayed, tap "**Continue**".
You can always manually enter your weight by selecting "**Enter my data manually**".

7. Answer the question: **I feel the same as usual / I don't feel as well as yesterday.**

15:10 19%

< Back Step 3/3 Symptoms X

I feel like usual

I feel worse than yesterday

8. If you feel worse than yesterday, select the symptom(s) or discomfort(s) you are experiencing.

The screenshot shows a mobile app interface for reporting symptoms. At the top, the status bar displays the time 15:10, signal strength, and 19% battery. The app header includes a 'Back' button, 'Step 3/3 Symptoms', and a close button. The main heading asks, 'Compared to yesterday, what new discomfort do you feel?' with a subtext '1 or more possible answers'. Below this, there are five selectable options, each with a radio button: 'I cough more', 'I'm more out of breath' (selected), 'I add a pillow to sleep', 'I'm more tired' (selected), and 'My ankles are more swollen'. A 'Nothing to report' option is also present. At the bottom, there is a 'Continue' button and an Android navigation bar.

15:10 19%

< Back Step 3/3 Symptoms X

Compared to yesterday, what new discomfort do you feel?

1 or more possible answers

☐ I cough more

☒ I'm more out of breath

☐ I add a pillow to sleep

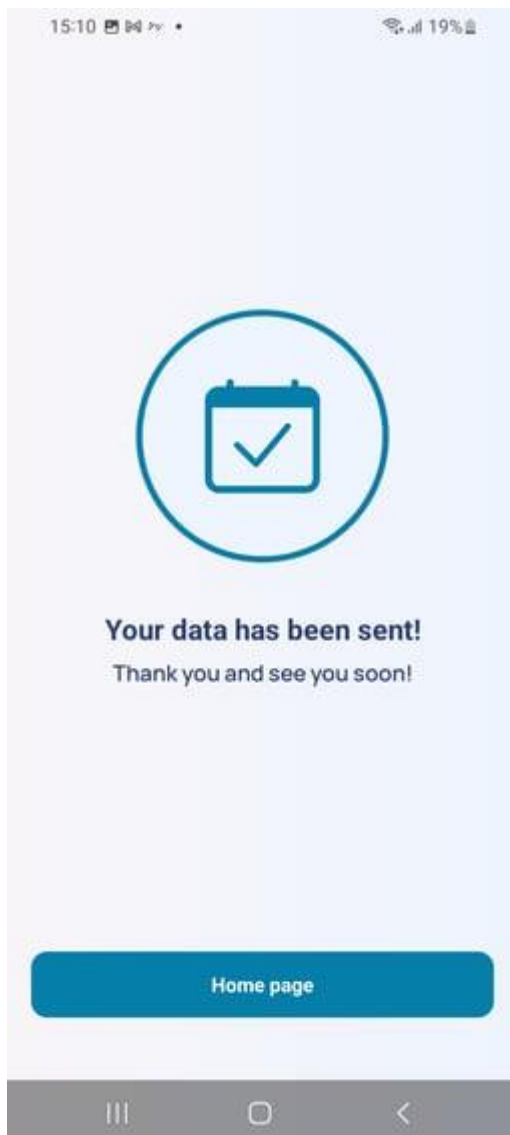
☒ I'm more tired

☐ My ankles are more swollen

Nothing to report

Continue

9. Tap "**Continue**". Your data has been successfully sent. Select "**Home screen**".

10. Turn off the tablet: press the power button for three seconds. A pop-up will appear: select "**Shut down**".

07. LAB RESULTS ENTRY:

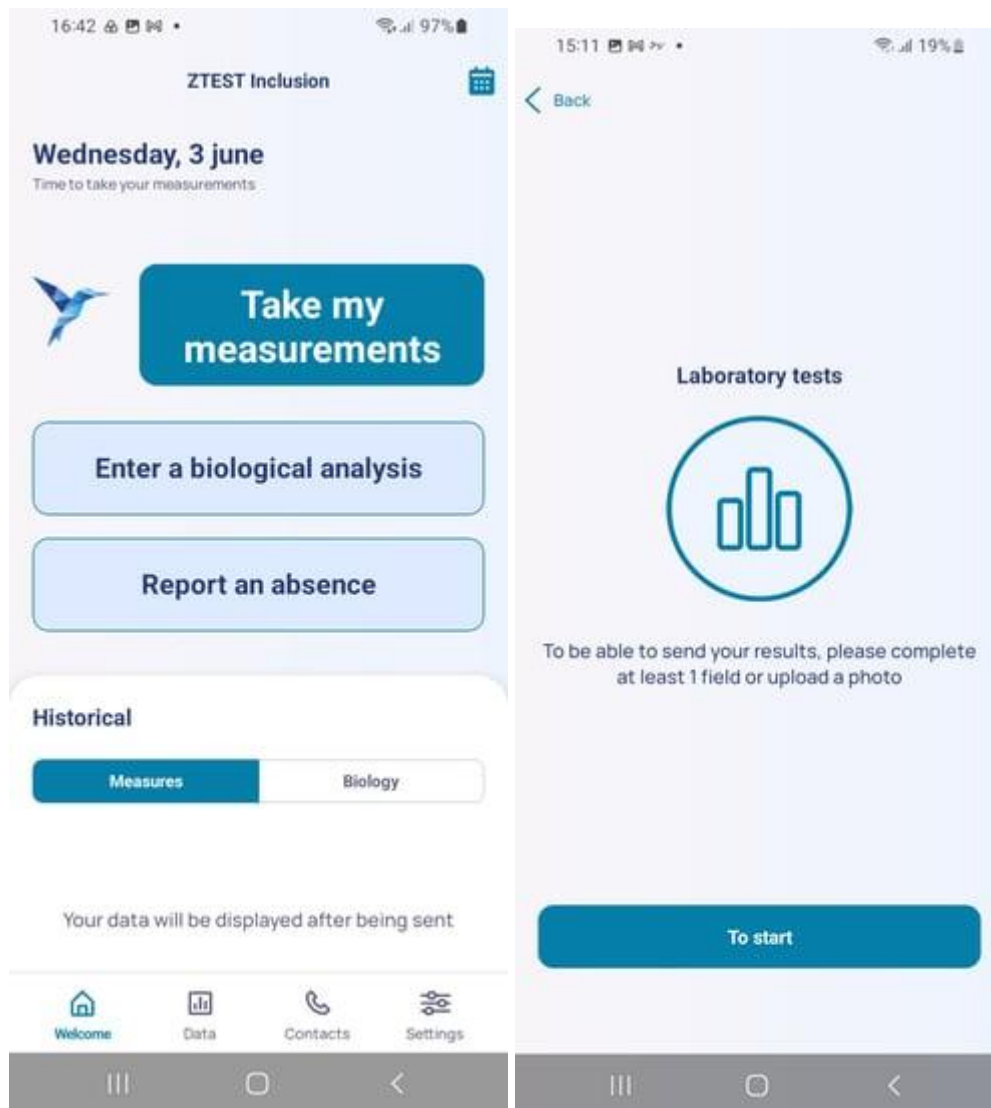
When your cardiologist prescribes lab tests, you can enter your results into your tablet as soon as you receive them. You can only enter one lab test report per day. In the application, you can enter the date of your test and scroll through the different tabs to fill in the values shown on your lab report.

Lab tests are divided into tabs:

Biochemistry, Cardiology parameters, Hematology, and a photo of the report to make entering your results easier.

Your lab data will appear directly on your cardiologist's platform, where they can review it and access a dated history of your results.

1. Upon receipt of your results, you can enter them into the NEWCARD application.
Tap: "+ **Lab tests**" then "**Start**".

2. Set the date of your analysis report. To do this, select the date directly with the key or click on the calendar button to open the calendar. After selecting the correct date, tap "**Continue**".

15:11

19%

< Back

Step 1/4
Biochemistry

✕

Analysis date

23/10/2025

📅

BNP

52

pg/mL

NTproBNP

pg/mL

Creatinine levels

mg/L

GFR (CKD-EPI)

ml/min/1.73 m²

🔄

🔊

⚙️

🔍

🔄

⋮

1

2

3

✕

4

5

6

OK

7

8

9

.-

0

,

|||

○

∨

📄

[Back](#)

Step 2/4
Haematology

Haemogram (blood count)

Erythrocytes or red blood cells

T/L

Haemoglobin

g/dL

Haemogram (white blood cell count)

Unit

/mm³

G/L

Leukocytes or white blood cells

/mm³

Neutrophils (PNN)

/mm³

Eosinophilic polymorphonuclear cells (PNEO)

/mm³

Basophilic polymorphocytes (BNP)

Continue

16:44 97%

< Back Step 3/4 Other X

Haematuria

☒ I don't know ☐ Yes ☐ No

Or enter the value

_____ hematies/mm3 v

Proteinuria over 24 hours

_____ mg/24h

Proteinuria on sample

_____ mg/g

Calcium

_____ mg/L

Continue

3. In the application, lab tests are divided into 4 steps to make data entry easier. You will find the following sections: **"Biochemistry"**, **"Cardiology parameters"**, **"Hematology"**, **"Photos"**, and **"Send"**. To move to the next section, tap **"Continue"**.

4. Each tab contains a list of fields you can complete. To scroll through the list, touch the screen and swipe up.

Fill in the data for which you have information on your lab report. Tap the field you want to complete. The numeric keypad will appear: enter your data by tapping the corresponding numbers. When you have entered all the data you have for a category, tap "**Continue**" to move to the next section.

The screenshot shows a mobile application interface for entering lab data. The top status bar displays the time 15:11, signal strength, and battery level at 19%. The app header includes a back arrow, the text 'Step 1/4 Biochemistry', and a close icon (X). The main form contains the following fields:

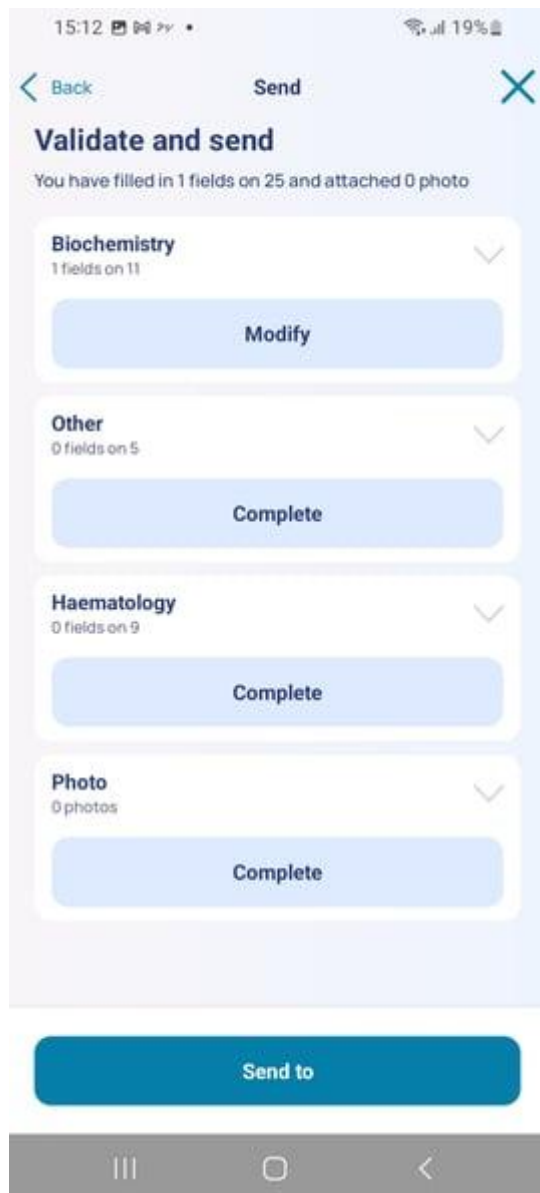
- Analysis date:** A date picker showing 23/10/2025.
- BNP:** A text input field containing the value 52, with the unit pg/mL displayed to the right.
- NTproBNP:** An empty text input field with the unit pg/mL displayed to the right.
- Creatinine levels:** An empty text input field with the unit mg/L displayed to the right.
- GFR (CKD-EPI):** An empty text input field with the unit ml/min/1.73 m² displayed to the right.

At the bottom of the screen, there is a numeric keypad with digits 0-9, a delete key (X), an OK button, a decimal key (.-), and a comma key (,). Above the keypad are several icons: a smiley face, a microphone, a settings gear, a magnifying glass, a person icon, and a three-dot menu.

5. Tap "**Add a photo**", then "**Take a photo**". Allow the application to access the camera. Position your results and tap "**Take a photo**" to capture the image. Once the photo is taken, confirm it by tapping "**Continue**". You can add more photos by tapping the "+". If you are not satisfied with the photo, tap "**Retake photo**" to try again.


6. Once the data is entered and the photo is uploaded, you can still edit the fields. If everything looks correct, tap "**Send**".

7. Your data has been sent. Select "**Home screen**".

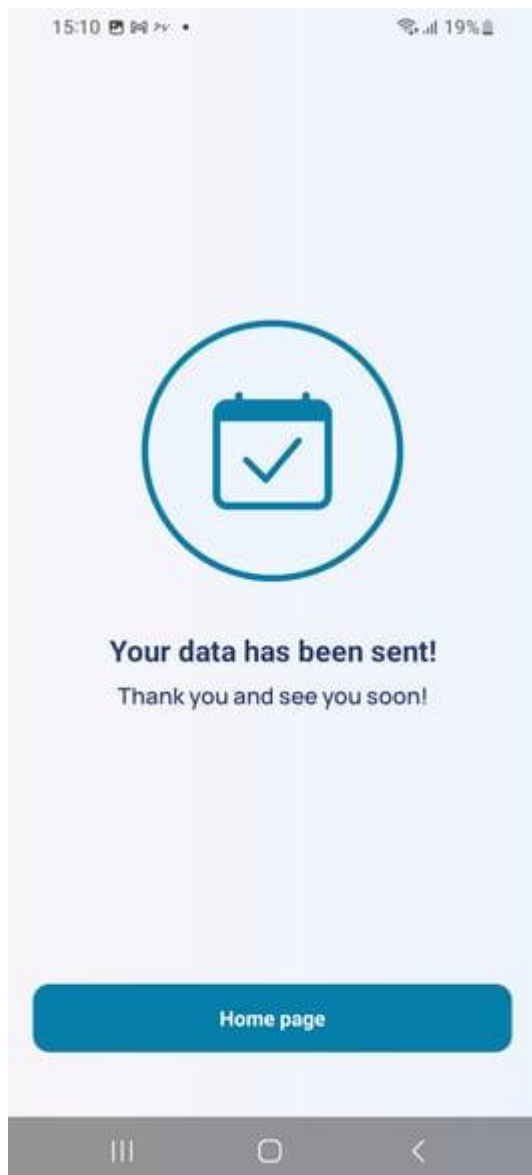Turn off the tablet: press and hold the power button for three seconds. A window will open: tap "**Power off**".

8. DATA VISUALIZATION:

You can access your clinical data history in the form of tables and graphs.
If you want to view your data, tap "**Data**" at the bottom of the home screen.

You can access your weight charts.

To return to the start, tap "**Home**".

Turn off the tablet: press and hold the power button for three seconds. A window will open:
tap "**Power off**".

9. HISTORY LOGS:

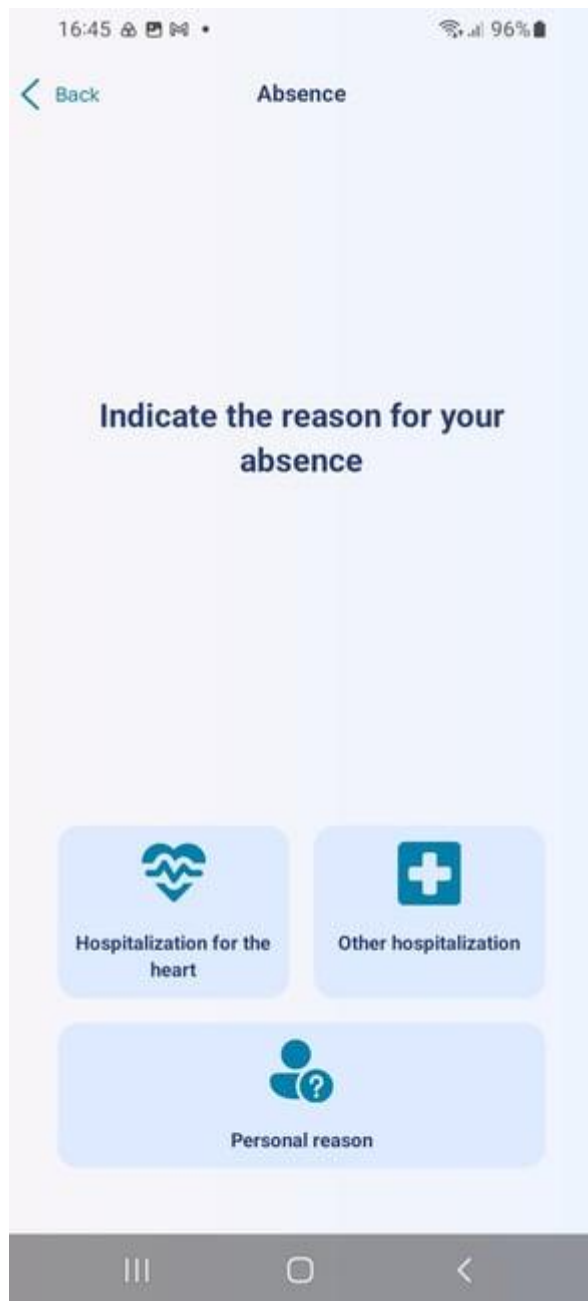
You can also view your measurement and lab test history by scrolling down on the home screen. In the "**Measurements**" tab, you will find the history of your weight, blood pressure, or heart rate.

In the "**Lab tests**" tab, you can access your lab report history. Select the date you want and tap the arrow to open the lab report you entered.

To return to the start, tap "**Home**". Turn off the tablet: press and hold the power button for three seconds. A window will open: tap "**Power off**".

10. NOTIFY OF AN ABSENCE:

1. On the home screen, tap "**Report an absence**".
2. Select the reason for your absence by tapping "**Personal reason**", "**Hospitalization for heart condition**", or "**Other hospitalization**".

3. Specify your absence by selecting your departure and return dates, then tap "**Send my absence**".

16:45 96%

< Back Absence

Please enter the dates of your absence

from

04/06/2026 

to

optional

Send my absence

4. Your absence has been successfully sent. Return to the home screen by tapping "**Home screen**".

11. MY HEALTHCARE TEAM :

In the "**Contacts**" tab on the home screen, you will find the Newcard support number and any other useful contacts. To return to the start, tap "**Home**".

12. SETTINGS :

In the "**Settings**" tab, you will find your connected devices, your phone number, your social security number, the option to change your password and use a PIN code, the Legal Terms of Use of the application, and the regulatory information regarding the device (the electronic label) and its version.

Manufacturer's address :
Newcard SAS
Parc du Moulin
236 avenue Clément Ader
59118 Wambrechies

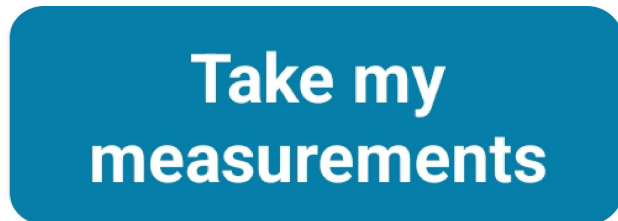
Enter weight and blood pressure manually -

Enter weight manually

This entry allows you to log your data manually in the event that one or all of your devices fail to connect to the app.

1. Select **"Take my measurements"**

Start the usual measurement procedure.


2. To take your weight, please follow the instructions displayed on the tablet.

3. If the error message **"We were unable to connect to the scale"**

Please try again in 5 seconds " is displayed, or if your weight is not displayed on the tablet, select :

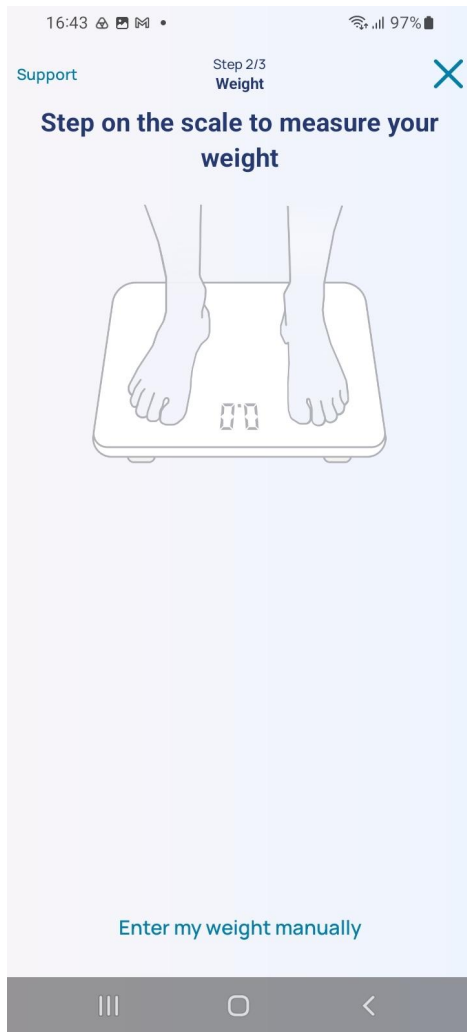"Enter my weight manually"

4. Enter your weight using the keyboard that appears.
5. Tap "Continue" to proceed to the next screen.
6. Answer the questions
7. Send your data
8. Turn off the tablet: press and hold the power button for three seconds. A window will open: tap "Power off".
Or close the mobile app.

Technical support-

Technical support

For any problem you encounter, please start by testing the various options listed below. Follow the solutions with the help of the symbol glossary.

Technical FAQ

Tablet or mobile app issues

The tablet will not turn on

Plug the tablet into the mains for several hours and turn it back on by pressing and holding the power button for **3 seconds**.

The following message appears: "Please allow the use of Bluetooth"

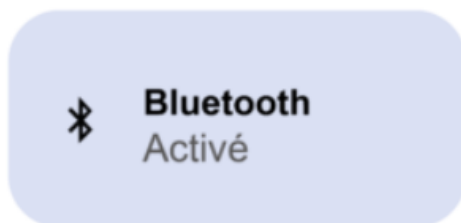
Bluetooth:


Tap "**OK**" to automatically activate Bluetooth, then tap "**Enable**".

You can verify that the Bluetooth icon is activated by swiping down from the top of the tablet screen.

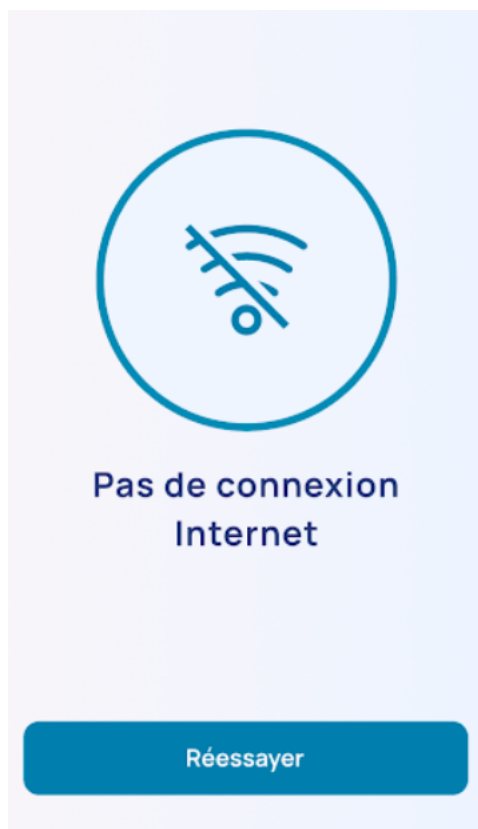The error message appears: "No internet connection."

Turn off your tablet by pressing and holding the power button for 3 seconds.

A window will open: Tap "**Turn off**".

Wait until the tablet is completely turned off.

Turn the tablet back on: press and hold the power button for 3 seconds.

Repeat the measurement procedure.

If the message is still displayed, please contact the Newcard support team.

The following message appears: "Turn on your location in settings"

Tap "**Turn on location**".

And enable the use of Location services in the settings. You can verify that the Location icon is activated by swiping down from the top of the tablet screen.

The following message appears: "Error loading data. Please try again"

Tap "**Reload**".

If it still does not work, restart the tablet: turn off your tablet by pressing and holding the power button for 3 seconds.

A window will open: Tap "**Turn off**".

Wait until the tablet is completely turned off.

Turn the tablet back on: press and hold the power button for 3 seconds.

If the problem persists, please contact the Newcard support team.

The tablet goes into "Reboot" or "Safe" mode.

Press and hold the power button **for 8 seconds** until the tablet restarts.

Issue with the scale

The scale displays the word "LO"

Turn the scale over and change the batteries (AAA batteries).

Repeat the measurement procedure as usual.

When measuring weight, an error message appears: "The application has stopped working."

Changing the scale batteries:

Turn the scale over and change the batteries (AAA batteries).

Repeat the measurement procedure as usual.

If the problem persists, please contact Newcard.

For any other technical questions not mentioned above, please contact us.

TERMS AND CONDITIONS-

TERMS AND CONDITIONS

LEGAL NOTICE FOR THE 'NEWCARD' APPLICATION

Patients are kindly requested to read the following Terms of Service for the 'Newcard' application carefully. We recommend that users refer back to these legal notices regularly, as they are subject to change.

ARTICLE 1 – GENERAL TERMS OF USE"

The purpose of the 'Newcard' application is to collect measurements taken by patients at home and transmit them via 3G or Wi-Fi to the platform developed by Newcard, which is accessible to healthcare professionals involved in telemonitoring. Its use is strictly reserved for patients suffering from heart failure who have been enrolled in the program by a physician and on whose device the application has been installed.

Particularly for the purpose of improvement, Newcard may modify or adapt the application. In such cases, an update by the patient may be required. The patient is required to perform this update as soon as possible.

Newcard also reserves the right to modify the application's terms of use at any time (by adding, changing, or removing elements). In this case as well, an update to the application by the patient may be necessary. Following each update, users are advised to review the legal notice.

ARTICLE 2 – INTELLECTUAL PROPERTY

This 'Newcard' application is published by Newcard, located at 236 avenue Clément Ader, 59118 Wambrechies, and is hosted by ATE (Avenir Télématique), 21 Avenue de la Créativité, 59650 Villeneuve-d'Ascq.

The elements created for this application are the exclusive property of Newcard or, where applicable, of third parties that have authorized their citation by the company. All elements (including trademarks, logos, images, graphics, icons, audio, software, and domain names) displayed on the 'Newcard' application are protected by applicable intellectual property laws and belong to Newcard or are subject to an authorization for use.

The user is granted only a right to use the information displayed (texts, logos, designs, images, etc.) on the application. This right is limited to personal and private use; the user has no right of reproduction, representation, modification, publication, or adaptation of all or part of the application's elements, except for a single copy intended strictly for private use, and subject to compliance with intellectual property rights and any other property rights mentioned. Any other use is subject to the prior agreement of Newcard. Consequently, any unauthorized use will constitute infringement, for which Newcard reserves the right to pursue legal remedies.

ARTICLE 3 – CONTACT INFORMATION

The owner and publisher of the 'Newcard' application is the company NEWCARD.

Newcard

236 Avenue Clément Ader

59118 Wambrechies,

France contact@newcard.io +33 (0)3 20 14 71 14

Registered with the Lille Métropole Trade and Companies Register (RCS) under number 821 766 961

SAS with a share capital of 3,511.00 euros

Managing Director: Pierre Emmanuel BONNET

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59650 Villeneuve d'Ascq, France

SAS with a share capital of 60,000 €

Lille Trade and Companies Register (RCS) No. 347 607 764

www.ate.info

+33 (0)3 28 800 300

Managing Director: Maxence ROUSSEAU

Questions or requests for information regarding the application may be sent to the company via the aforementioned postal or email addresses, or by phone using the contact details provided.

ARTICLE 4 – APPLICABLE LAW

The 'Newcard' application is an application created in France. Any dispute arising in connection with its use shall be governed by French law, and the French courts shall have exclusive jurisdiction.

For any questions regarding personal data or liability, users are invited to read the application's Terms of Service, which they have accepted.

TERMS OF SERVICE FOR THE 'NEWCARD' APPLICATION Please find attached the Terms of Service that you accept / accepted during your first log-in to the application. For any withdrawal, please contact NEWCARD using the contact details provided.

Between NEWCARD, a Simplified Joint-Stock Company (SAS) registered under identification number 821 766 961 with the Lille Métropole Trade and Companies Register, having its registered office located at 391 avenue Clément Ader, 59118 Wambrechies, represented by its legal representative (hereinafter referred to as the 'Service Provider' or 'NEWCARD')

And the patient, defined herein as a user of the Telemonitoring Software prescribed by their physician in accordance with the legal and regulatory requirements of the Social Security

(hereinafter referred to as the 'Patient'). NEWCARD and the Patient are hereinafter collectively referred to as the 'Parties' or individually as a 'Party'.

1. ACCEPTANCE AND UPDATES

Important : Any use of the Logiciel in any capacity whatsoever implies the unreserved acceptance by the end user (the Patient) of these TOU.

1.1 The Patient acknowledges that he/she has read, understood and accepted these TOU without limitation or condition, initially by giving his/her free and unreserved consent to his/her physician to be tele-surveilled and use the "Newcard" Application, then by accepting these TOU on the Application (i.e. the "System" or the "Software"). These TOU constitute the complete and exclusive agreement between the Patient and NEWCARD concerning the use of the Logiciel.

1.2 NEWCARD reserves the right to change, modify all or part of these TOU. The Patient is informed of these modifications and of the date of updating of the TOU.

If the Patient does not accept these GTU or any updated version of these GTU, he must not use the System.

1.3 The Patient hereby warrants and confirms that he/she is of legal age to consent to these TOU and that he/she is fully capable of (i) concluding the contract that these TOU constitute, and (ii) complying with them.

2. PURPOSE

2.1 NEWCARD makes the Application available to Patients under the Decree of June 22, 2023, concerning the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No. 2021-1754 of December 23, 2021, on social security financing for 2022.

2.2 NEWCARD makes the Application available to the Patient and grants them a non-exclusive, personal, and non-transferable license to use the System solely within the French territory.

2.3 The System offers the following main features:

- Weight tracking using a connected scale;
- Blood pressure and heart rate measurement using a connected blood pressure monitor;
- Symptom reporting through a questionnaire consisting of five (5) questions;
- Information logging for biological data;

- Application on a digital tablet or smartphone allowing the retrieval and visualization of measurements taken, as well as notification of absence;
- A 3G system for sending measurements, particularly for patients who do not have an internet connection;
- An algorithm allowing data processing and the triggering of alerts in the event of a variance;
- A telephone service to answer technical questions, on business days from 9:00 AM to 5:00 PM.

3. FREE SERVICE AND CONNECTION CONDITIONS

3.1 The provision and use of the System are free of charge for eligible patients under the Decree of June 22, 2023, concerning the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No. 2021-1754 of December 23, 2021, on social security financing for 2022, and to whom the telemonitoring service has been medically prescribed.

3.2 The Patient acknowledges that they possess the necessary means and skills to use the System. Consequently, the Patient accepts and understands that the performance of the System may be affected by the means they choose to use. Furthermore, if the Patient uses their own internet connection to send their data, they are responsible for their internet subscription, and NEWCARD shall not be held liable for the billing terms of any third-party operator.

3.3 In accordance with Article L.221-28 of the French Consumer Code, the Patient does not benefit from a right of withdrawal once they validate their registration to the System, thereby accessing the System's digital content. In any event, the Patient may withdraw their consent at any time by contacting NEWCARD using the contact details provided to request the closure of their account on the System.

4. OBLIGATIONS OF NEWCARD

4.1 The Service Provider makes the System available at the Patient's home and offers telephone support in the event of a technical issue, available from Monday to Friday from 9:00 AM to 5:00 PM.

4.2 The Service Provider undertakes to host the Patient's data within the System through a hosting provider certified to store health data (HDS certified hosting provider).

4.3 The Service Provider undertakes to comply with the provisions of Regulation (EU) 2016/679 on the protection of natural persons with regard to the processing of personal data and on the free movement of such data, and repealing Directive 95/46/EC (General Data Protection Regulation).

4.4 The Service Provider undertakes, within the framework defined by these Terms of Service, to use all reasonable means to ensure the performance and continuity of access to the System. However, the Patient accepts and acknowledges that maintenance and update operations on the System may occur, and the Service Provider will use its best efforts to notify the Patient in advance.

4.5 The Service Provider undertakes to comply with the applicable regulatory provisions relating to its status as a regulated Medical Device and, as such, bears the CE marking.

5. MEDICAL DISCLAIMER

5.1 The Software provides and publishes informative content based on the results of the measurements taken by the Patient and is solely intended to assist them and provide information required by the System for the proper conduct of telemonitoring. NEWCARD is not a medical organization or a healthcare professional. NEWCARD offers a System for transmitting and organizing information sent by the Patient. In this regard, the System informs physicians of the data sent. None of the content on the Software shall be interpreted to mean otherwise. Consequently, the information accessed on the Software shall not be considered a substitute for (a) a personal and professional medical consultation provided by a physician, and/or (b) a diagnosis, interpretation, or evaluation made by the physician either remotely or in person.

5.2 THIS SYSTEM IS NOT AN EMERGENCY SYSTEM. Its purpose is the daily monitoring of data sent in order to notify the physician in the event of a significant variance in the data. Telemonitoring follow-up does not exempt the Patient from their usual care routines. In the event of any medical difficulty or emergency, entirely left to their own judgment, the Patient must contact healthcare professionals as they normally would (for example, their general practitioner or emergency medical services).

5.3 NEWCARD shall not be held liable:

- If data fails to reach it, for any reason whatsoever, or if the data received is illegible or impossible to process;
- If the user is unable to access or use the application, for any reason whatsoever;
- If, for any reason whatsoever, the connection is interrupted.

6. PATIENT OBLIGATIONS

6.1 The Patient undertakes to allow NEWCARD to create an account for them and to train them on how to use the System if necessary. They then undertake to take their measurements regularly and in accordance with the recommendations made by Newcard for their telemonitoring follow-up. This is an essential condition for the proper functioning of the telemonitoring system.

6.2 The Patient, as a user, acknowledges that they possess the necessary means and skills to use the Software. Consequently, the Patient accepts and understands that the performance of the Software may be affected by the chosen connection means.

6.3 The Patient undertakes to make strictly personal use of the equipment provided to them, in accordance with its intended purpose. They shall follow the instructions provided during installation and/or the information included in the user guide. They are aware that they have custody and responsibility of the equipment, and notably the tablet equipped with the Application. They are also aware that the trend curves of their data can be visualized when connected to the internet or a 3G server. In this respect, they are solely responsible for any use of the tablet and the Application by third parties, and must therefore take the necessary security measures. On their own initiative, they may notably lock the tablet or smartphone on which they use the Application with a confidential code of their choice. In the event that the tablet provided by Newcard is locked by a confidential code, they must imperatively remove this code before returning the equipment to Newcard.

6.4 The Patient undertakes to use their best efforts to enter their data into the System in a reliable, complete, accurate manner, and in accordance with the formats agreed upon between the Parties. In case of doubt or difficulty, the Patient may contact Newcard for assistance with data entry.

6.5 The Patient undertakes to comply with the technical prerequisites and hardware configurations recommended by the Service Provider.

6.6 At the end of their telemonitoring follow-up or upon termination of the Contract by either Party, for any reason whatsoever, the Patient or their beneficiaries (ayants droit) undertake to return the equipment to NEWCARD within the following 30 days, by sending it to the address provided to them. The Patient also undertakes to return the equipment to NEWCARD in the event of defective equipment or any other reason requiring a kit replacement.

6.7 The Patient undertakes to comply with the provisions of these Terms of Service and the requirements of the guidelines set by the Social Security.

6.8 The Patient undertakes to comply with applicable laws and regulations in force.

7. COMMON OBLIGATIONS

7.1 Each Party undertakes to inform the other Party of any event or information brought to its knowledge that would be likely to have any effect whatsoever on the Software and these Terms of Service.

7.2 Each Party undertakes to respect the confidentiality of these presents and not to harm the image and reputation of the other Party.

8. INTELLECTUAL PROPERTY

8.1 NEWCARD remains the sole owner of all its intellectual and material property rights regarding trademarks, photographs, computer programs, distinctive signs, know-how, methods, and all documents used or implemented during the use of the Software, and notably its algorithm (all such rights and elements hereinafter referred to as the "Elements").

8.2 The Patient undertakes not to infringe upon, or challenge in any manner whatsoever, the Service Provider's rights over the Elements belonging to it.

8.3 The Contract does not imply any transfer of ownership over the Software, and notably over the data, methods, or development tools for which the Service Provider holds proprietary or licensing rights and which may have been made available to the Patient for the purposes hereof.

8.4 The Patient undertakes not to make any modification to the Software. The Patient acknowledges that the Software is protected by intellectual property rights and declares that they have not entered into this contract with the intent to infringe upon them in any manner whatsoever. In particular, the Patient undertakes not to copy it, disclose it in any way whatsoever, and not to commit or allow any action to be committed that is likely to infringe upon these rights.

9. LICENSE FOR USE

9.1 The Service Provider hereby grants to the Patient, who accepts it, a personal, non-transferable, non-assignable, and non-exclusive license to use the Software for a term of 2 years, tacitly renewable. This license shall terminate at the end of their telemonitoring follow-up.

9.2 The granting of the right of use, as provided for in the Contract, is consented solely to the Patient. This grant is validly attributed solely for the French territory from the moment a session is opened on the Software.

9.3 The Patient is prohibited from any type of exploitation not explicitly authorized by these Terms of Service, and notably from:

- Making a backup copy for any reason other than preserving the use of the Software;
- Correcting or having a third party correct any potential errors or bugs in the Software without the prior written consent of the Service Provider;
- Granting a loan or making the Software or its documentation available by any means whatsoever, including via the Internet;
- Adapting, arranging, or modifying the Software.

10. ENTRY INTO FORCE / TERM

10.1 This document enters into force as of the date of acceptance of these Terms of Service.

10.2 The duration of the Contract is 2 years.

10.3 The Contract shall be tacitly renewed for successive periods of two years, except in the event of (a) termination by either Party notified by registered letter with acknowledgment of receipt subject to one (1) month's notice, or (b) the end of the medical telemonitoring follow-up and non-renewal of the prescription by the Cardiologist.

11. CONFIDENTIALITY

11.1 The term "Confidential Information" hereinafter refers to any information, whether materialized or not, that is strategic, financial, technical, or commercial. This confidential nature may notably result from the presence of any appropriate marking, from the very nature of the document, or from an express oral indication.

11.2 The Parties undertake to ensure that Confidential Information is kept confidential by taking all useful and reasonable security measures, and used solely for the purposes hereof.

11.3 The provisions of this article shall remain valid throughout the term of the Contract and for 4 years thereafter.

12. PERSONAL DATA

12.1 Identity of the Data Controller

The collection and processing of personal data are carried out by the Company Newcard, responsible for the application made available to patients. Newcard is domiciled at the following address: 391 avenue Clément Ader, 59118 Wambrechies. It can be contacted at +33 (0)3 20 14 71 14 or by email at info@newcard.io.

Newcard has appointed a Data Protection Officer (hereinafter "DPO"), duly declared to the French National Commission for Information Technology and Civil Liberties (hereinafter "CNIL"). The DPO can be contacted at +33 (0)3 60 85 06 86 or by email at mes-donnees-personnelles@newcard.io.

12.2 Purposes of Personal Data Processing

The data collected is processed for specific purposes. Consequently, it cannot be used for purposes other than those specified in these Terms of Service. In this case, these purposes are of several types:

- Conducting the telemonitoring of patients suffering from chronic pathologies;
- Providing therapeutic support; and
- Conducting analyses and data studies on patient populations. The objective is to contribute to scientific research on chronic pathologies and their management, telemonitoring, and its medico-economic benefits.

12.3 Legal Basis for Processing

The collection and processing of data are carried out in accordance with the provisions of Regulation (EU) 2016/679 of the European Parliament and of the Council of 27 April 2016 on the protection of natural persons with regard to the processing of personal data and on the

free movement of such data (GDPR), and Law No. 78-17 of January 6, 1978, relating to information technology, files, and civil liberties. The legal basis for processing is the consent of the data subject.

12.4 Categories of Processed Data

Only the data necessary for the patient's telemonitoring within the framework of the tele-follow-up application is required.

This includes the patient's identity data (last name and first name, gender, date of birth), their social security number, their daily vitals (weight, height, blood pressure, heart rate, symptom questionnaire), connection data, and the software usage history.

12.5 Recipients of Personal Data

The collected information is secured and hosted by an HDS certified hosting provider ("Hébergeur de données de santé"). Our data host is ATE, whose contact details are as follows:

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59650 Villeneuve d'Ascq

<https://ate.info> / contact_web@ate.info / +33 (0)3 28 80 03 00

Several persons, whether belonging to Newcard or not, are authorized to access the patient's personal data:

- The cardiologist and their medical team have access to the patient's personal data (sensitive and non-sensitive) in order to ensure their telemonitoring;
- Internal healthcare professionals at Newcard also have access to the data (sensitive and non-sensitive) to ensure the efficiency of the patient's follow-up; and
- Newcard's administrative staff also has access to the patient's non-sensitive personal data (identity, contact details, etc.).

12.6 Retention Period of Personal Data

All data collected and processed by Newcard within the framework of the application is retained:

- For patients followed by a private practice cardiologist: 5 years after the end of their follow-up;
- For patients followed by a healthcare institution (hospital or private clinic): 20 years after the end of their follow-up. At the end of the retention period, the data will be irreversibly anonymized.

12.7 Rights of Data Subjects

Regarding the collection and processing of personal data, data subjects possess numerous rights necessary to guarantee fair and transparent processing. To exercise these rights, they may contact Newcard at the contact details mentioned in paragraph 12.1. Thus, they have a right to:

- **Access to personal data:** The data subject has the right to obtain from Newcard confirmation as to whether or not personal data concerning them is being processed, and, where that is the case, access to said data.
- **Rectification of personal data:** The data subject has the right to obtain from Newcard the rectification of inaccurate and/or incomplete personal data.
- **Erasure of personal data:** The data subject has the right to obtain from Newcard the erasure of personal data concerning them without undue delay.
- **Restriction of processing:** The data subject has the right to obtain from Newcard the restriction of processing of their personal data.
- **Objection to processing:** The data subject has the right to object, at any time, to the processing of personal data concerning them.
- **Data portability:** The data subject has the right to receive the personal data concerning them, which they have provided to Newcard, in a structured, commonly used, and machine-readable format.
- **Withdrawal of consent:** The data subject has the right to withdraw their consent to the processing of their personal data at any time, without affecting the lawfulness of processing based on consent before its withdrawal.
- **Lodging a complaint with a supervisory authority:** The data subject has the right to lodge a complaint with the competent supervisory authority. They may address their remarks to the Commission Nationale de l'Informatique et des Libertés (CNIL), whose contact details are as follows: Commission Nationale de l'Informatique et des Libertés 3 Place de Fontenoy – TSA 8071575334 Paris Cedex 07

12.8 Transfer Outside the European Union

The data collected and processed within the framework of telemonitoring is hosted by ATE – an HDS certified data host – based in Villeneuve d'Ascq.

Our host certifies that the data is hosted within the European Union and that no transfer outside the European Union is carried out. Newcard undertakes to provide any interested person, upon request, with the hosting certificate issued by ATE.

13. TERMINATION - EXPIRATION

13.1 Under the conditions provided for in Article 10.3, the Parties may terminate the Contract. This termination shall take effect 30 calendar days from the receipt of the termination notice.

13.2 The prescription by the Physician of the Newcard service under the Decree of June 22, 2023, concerning the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No.

2021-1754 of December 23, 2021, on social security financing for 2022, is an essential condition of these Terms of Service. In the event of the end of the medical telemonitoring follow-up, the Contract shall terminate automatically, except for authorized out-of-framework follow-ups, which require the express agreement of Newcard.

13.3 In the event of non-performance of their obligation to return the equipment, the Patient will be served a formal notice to pay the sum of two hundred (200) euros to NEWCARD, representing the buyback price of the equipment, by check or bank transfer using the details provided to them by mail.

13.4 Upon expiration or termination of the Contract for any reason whatsoever, the obligations of the Parties shall survive regarding intellectual property and copyright, confidentiality, and those clauses which by their nature must remain in force shall remain in force, such as the "Liability" clause.

14. SUSPENSION – TERMINATION OF ACCESS

14.1 The Service Provider may automatically suspend access to the Software and remove the Software from online availability, without compensation to the Patient and without prior formal notice, in the following cases: (1) upon the Patient's request, (2) due to non-compliance with the Patient's obligations, (3) the Patient's death, (4) discontinuance of the Software, (5) when the Patient's actions endanger the Software in any manner whatsoever, (6) in the event of a material breach of this Contract, (7) in the event of the end of a follow-up semester and non-re-inclusion of the patient by their cardiologist. This decision shall be brought to the Patient's attention by the Service Provider within a reasonable timeframe.

14.2 In the event of suspension of access or termination of the Contract, the Patient has the possibility to extract their data in accordance with applicable regulations.

15. LIABILITY

15.1 The Service Provider shall not be held liable in any manner whatsoever for the consequences resulting from (i) the Patient's lack of diligence in taking their vitals, (iii) the incorrect use of the Software by the Patient, (iv) the use by any third party of the equipment under the sole custody of the Patient, (v) any delay, inaccuracy, or malfunction due to a failure, fault, negligence, or omission of the Patient, their physician, their subcontractors, or any third party over whom the Service Provider has no power of control or supervision.

15.2 Neither Party shall be held liable for the consequences of indirect damages. In particular, data losses are considered indirect damages.

15.3 It is up to the Patient to inform the Service Provider of any defect or non-conformity observed, and to provide proof of the Service Provider's failure.

15.4 The Service Provider shall not be held liable in any manner whatsoever for (v) total or partial unavailability to access the Software, notably for technical maintenance operations or

updating of published information, (vi) viral attacks, (vii) incomplete or erroneous information provided by a user, (viii) the processing or lack of processing of alerts by the Physician.

16. FORCE MAJEURE

Neither Party shall be held liable for any breach of its obligations under the Contract if such breach results from a situation constituting force majeure as defined by the case law of the French Cour de Cassation on the date of its occurrence. Each Party shall immediately inform the other Party of the occurrence of a force majeure event likely to prevent it from executing its obligations. The Parties shall then endeavor to take appropriate measures to mitigate the consequences of the event. However, if the event persists beyond 30 calendar days, the Contract may be terminated by the most diligent Party, without compensation.

17. MISCELLANEOUS

17.1 If any of the clauses of the Contract were to be declared null and void, the remaining provisions of the Contract shall remain in full force and effect.

17.2 The failure of either Party to invoke a breach by the other Party of any of the obligations referred to in the Contract shall not be construed for the future as a waiver of its right to invoke said breach or to enforce the obligation in question.

18. GOVERNING LAW – JURISDICTION

18.1 The Contract is governed by French law.

18.2 UNLESS CONTRARY TO PUBLIC POLICY PROVISIONS, ANY DISPUTE RELATING TO THE APPLICATION, PERFORMANCE, OR INTERPRETATION OF THE CONTRACT SHALL, BY EXPRESS AGREEMENT, BE BROUGHT BEFORE THE COURTS OF LILLE.

Contact us

If you have any questions, please contact NEWCARD, a simplified joint-stock company, at **03.20.14.71.14**

The head office is located at 236 Avenue Clément Ader 59118 Wambrechies.

MPC-NTU-PAT-ENO-01

This user manual MPC-NTU-PAT-ENO-01, created on June 24, 2026, may be subject to change depending on updates to the platform. If this is the case, we will be sure to inform you of any changes.

The patient must be informed that remote monitoring does not constitute emergency care and that recorded data or alerts are not read and interpreted instantly, but only during the days and hours set by the remote monitoring operator. The patient is therefore informed that in case of emergency, he must contact an emergency call number. -

<https://www.legifrance.gouv.fr/jorf/id/JORFTEXT000047720200>

The "Newcard" application is a telemonitoring device for chronic heart failure, allowing you to regularly send your measurements such as weight and lab results to your healthcare professionals. The purpose of this system is the potential early detection of cardiac decompensation.

NEWCARD makes the Application available to Patients pursuant to the Decree of June 22, 2023, regarding the registration of medical telemonitoring activities on the list provided for in Article L. 162-52 of the French Social Security Code, on the basis of Article 36 of Law No. 2021-1754 of December 23, 2021, on the financing of social security for 2022.

Head office: Wambrechies - 821 766 961 - Registered with the Trade and Companies Register of Lille Métropole B.

Creation date: June 24, 2026 - Image: Newcard

If needed, a printed version can be provided upon request at the following address: <https://newcard.io/contacts/> or via email at cardio@newcard.io