


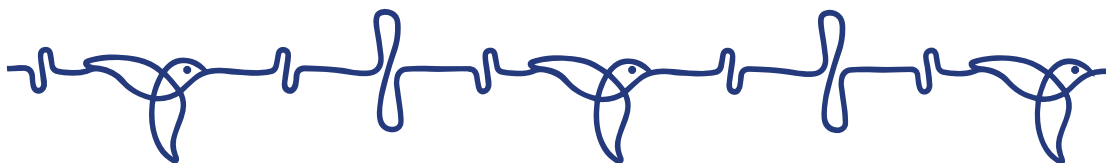

1 MINUTE FOR *my heart*

QUICK USER GUIDE



TAKE YOUR MEASUREMENTS

In 1 minute tops

1




Open the tablet
Select the Newcard app

2 Tap to select

Take my measurements

4 Send my blood pressure

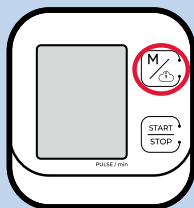

Press the button



5 Weight yourself




(Scale should be placed
on a hard and stable surface)

INDICATE AN ABSENCE


Tap to select

Notify an absence

on the
homepage

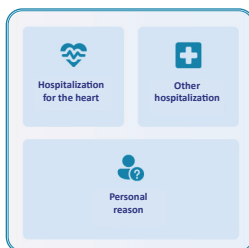

**Indicate the reason
for the absence**

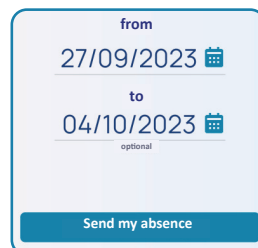

**Choose the date of
the absence and tap**

Send my absence

3 Take your blood pressure correctly

In 4 steps




1. Sit down and place the cuff around your **left arm**.
2. The **red band** should be placed inside the **crease of your elbow**.
3. Place your **left arm on a table**, with the **palm facing up**.

4. Press start


START
STOP

6

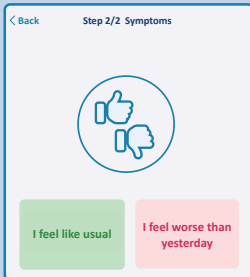

Answer the questions

7

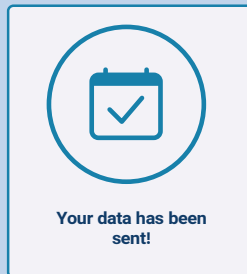

Send your data

8

Your data is monitored by your Newcard nurse and your cardiologist.



OUR ADVICE

- Make sure the batteries are inserted in both the scale and the blood pressure monitor.
- To take your blood pressure, sit comfortably in a calm room.
- Do not unroll/unfold the cuff completely, slide your arm into it.
- Place the cuff on your arm, at the same level as your heart.
- The red mark should be in the crook of your elbow.

For the first measure, don't hesitate to ask your relatives, your caregiver, or your neighbour.

YOUR THERAPEUTIC SUPPORT


One mandatory session of approximately 15 minutes
by phone, with additional sessions available for:

Getting to know your condition better,

**Learning the appropriate behaviors
during your remote monitoring,**

Getting involved in your care journey.

Contact your cardiologist or Newcard to learn more.

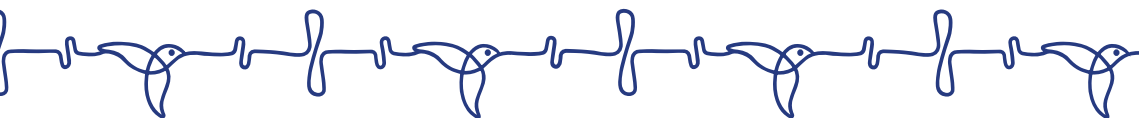

CONTACTS



Phone number
03 20 14 71 14



SMS
07 57 90 27 24



E-mail
cardio@newcard.io

In case of emergency
call 15

FIND OUT MORE



[http://26570142.hs-sites-eu1.com/
complete-user-guide-1](http://26570142.hs-sites-eu1.com/complete-user-guide-1)

**We kindly ask that you keep the original packaging
to transport the kit or return it to us.**

This Class I medical device is a health product that bears the CE marking under this regulation (2019).
Please read the instructions in the user manual carefully.

Manufactured by NEWCARD SAS - Last modification : 28 january 2025 - Printed by : Pixartprinting
Do not litter or discard in public areas – Reference : MPC-NTU-PAT-ENG-00



Parc du Moulin
391 Av. Clément Ader
59118 Wambrechies
www.newcard.fr